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*Assembly instructions  
and user guidelines*

Manual version : updated on **10/2025**

---

**Solide, polyvalente, réversible, mobile**

*Sturdy, multifunctional, reversible and mobile*

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**You have just received  
your Universal Hand  
Table in its transport  
case.**

Asclepios France thanks  
you for your interest in this  
equipment.

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Please take a moment to read the following instructions in full and watch the video on our website, which shows you how to assemble the table step by step.

The entire process takes just a few minutes and can be completed by one person, except for laying the case flat and turning the table over, which require assistance from a second person.

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Operations that require assistance from a second person are indicated by the following icon:

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## PART 2: DISCOVERING THE OPTIONS

1. Small extension .....	page 11
2. Medium extension .....	page 12
3. Large extension .....	page 13
4. The drape holder .....	page 14
5. The arm holder .....	page 15
6. Support arm extension and arthroscope holder .....	page 16
7. Arthroscopy ancillary instruments .....	page 17
8. The horizontal traction kit .....	page 18
9. Assistance using the Alsatian traction .....	page 19
10. Mayo stand .....	page 20
11. Articulated arm and screen/tablet/mirror support .....	page 21
12. Light prop .....	page 22
13. Lower X-ray shutter panels .....	page 23
14. Magnets .....	page 25
15. Dedicated stool .....	page 26

## PART 3: OPENING THE CARRYING CASE

Opening and content .....	page 27
---------------------------	---------

# CONTENTS

## PART 4: STEP BY STEP ASSEMBLY

Step 1: Wheeled feet assembly.....	page 30
Step 2: Table height adjustement .....	page 34
Step 3: Installing and storing the extensions.....	page 36
Step 4: Installation of the support arm and its accessories for vertical traction .....	page 37
Step 5: Arm support.....	page 39
Step 6: Fitting the remaining accessories .....	page 42
Step 7: Assembly of arthroscopy accessories .....	page 43
Using a Finochietto handle.....	page 46
Use of Japanese finger traps .....	page 47
Important information: General guidance for surgical traction .....	page 48

## PART 5: THE MAYO STAND

Assembling the Mayo stand .....	page 50
Using the Mayo stand in sterile conditions .....	page 53
Use of the Mayo stand in NON-sterile conditions .....	page 54


## PART 6: APPENDICES

Appendix 1: Description of the paretts/accessories for arthroscopy with a Finochietto handle....	page 56
Appendix 2: Description of the parts/accessories for use with Japanese finger traps .....	page 58
Appendix 3: Using horizontal traction.....	page 60
Appendix 4: Fixation of the patient's fingers: Japanese finger cots/Hand of Finochietto.....	page 66
Appendix 5: Increasing the shaft height .....	page 68
Appendix 6: Ratchet mechanism.....	page 69
Appendix 7: Traction known as the "Alsatian model" .....	page 70
Appendix 8: On-board lighting .....	page 71
Simplified Universal Hand Table (for care and consultations) .....	page 73

# LA TABLE À MAIN UNIVERSELLE - INOX - BLOC UNIVERSAL HAND TABLE – STAINLESS STEEL – OPERATING ROOM

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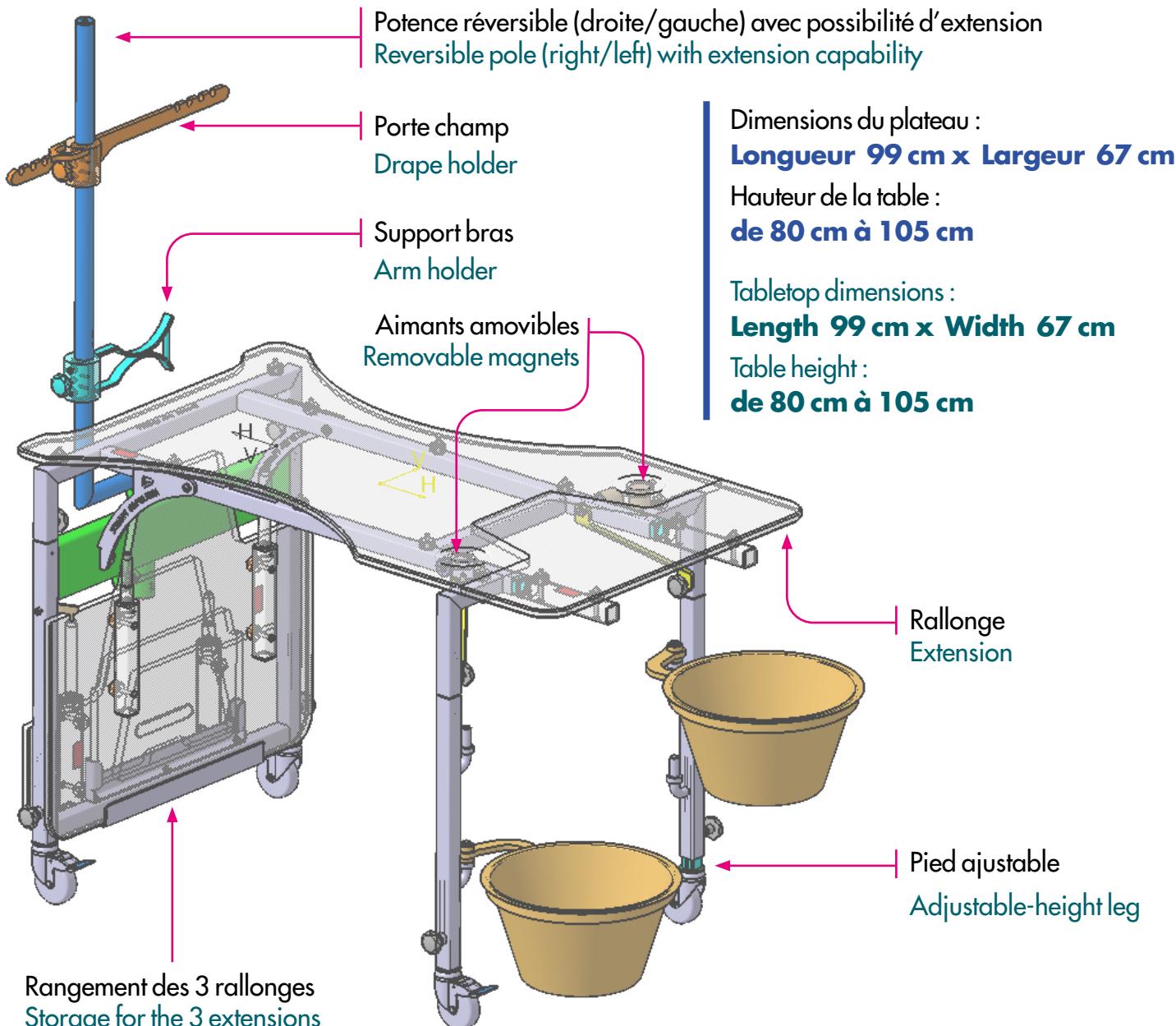
- Robust equipment.
- Long-term compatibility.
- New features can be retrofitted to earlier models of the Universal Hand Table.
- Optional updates available.
- Preventive maintenance available for intensive use.

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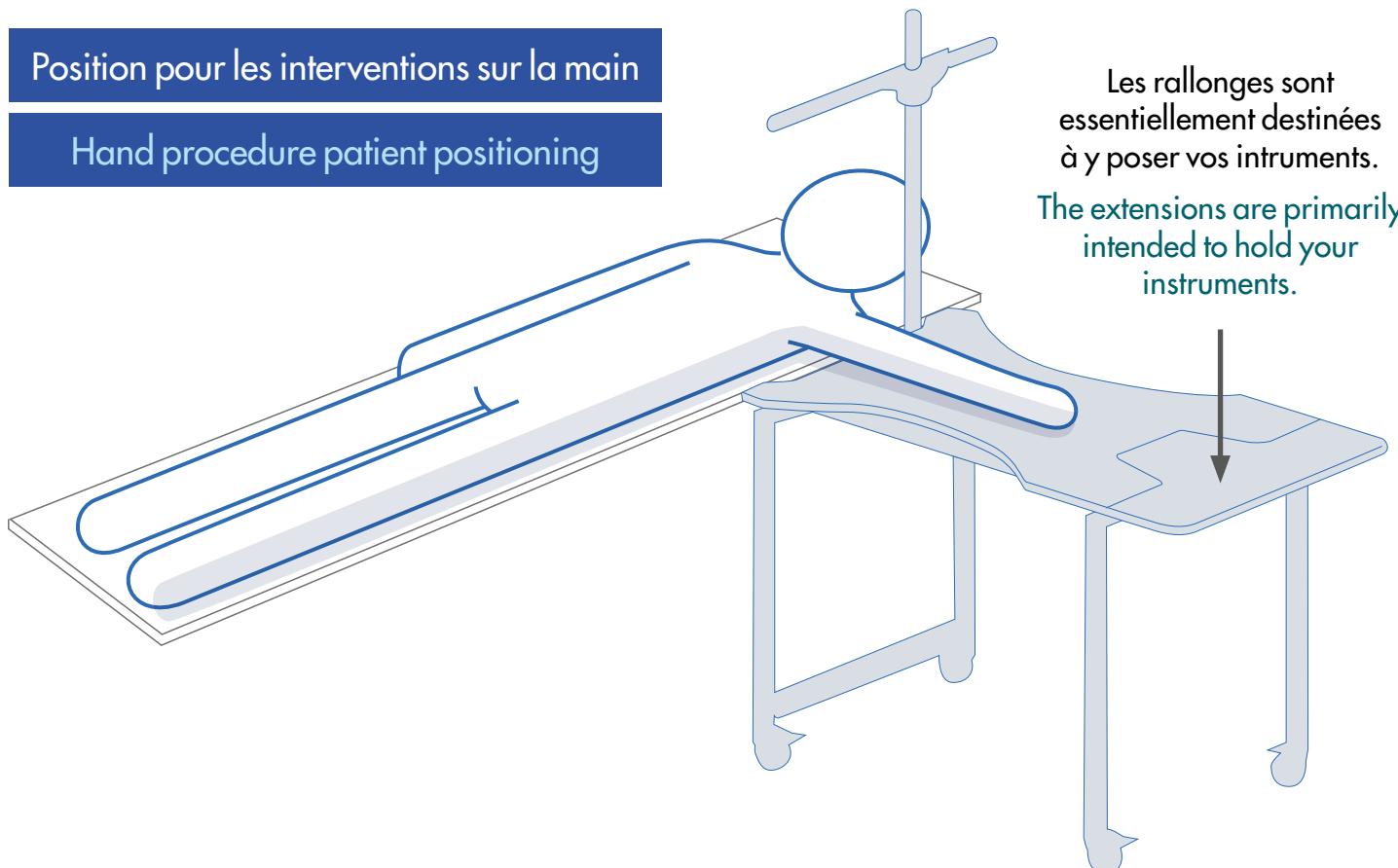
Hand procedure patient positioning

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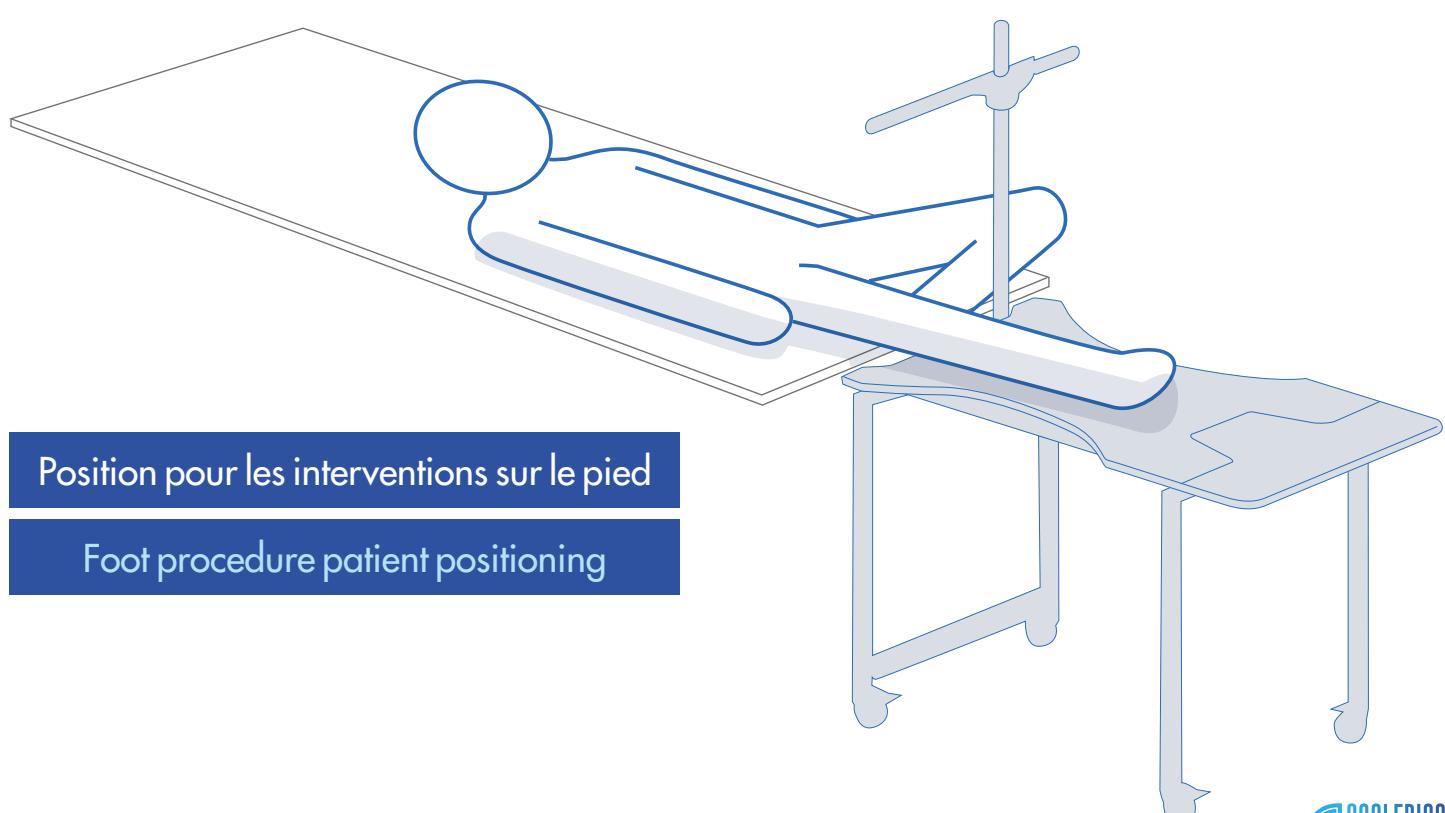Foot procedure patient positioning

# LA TABLE À MAIN UNIVERSELLE - INOX - BLOC UNIVERSAL HAND TABLE - STAINLESS STEEL - OPERATING ROOM

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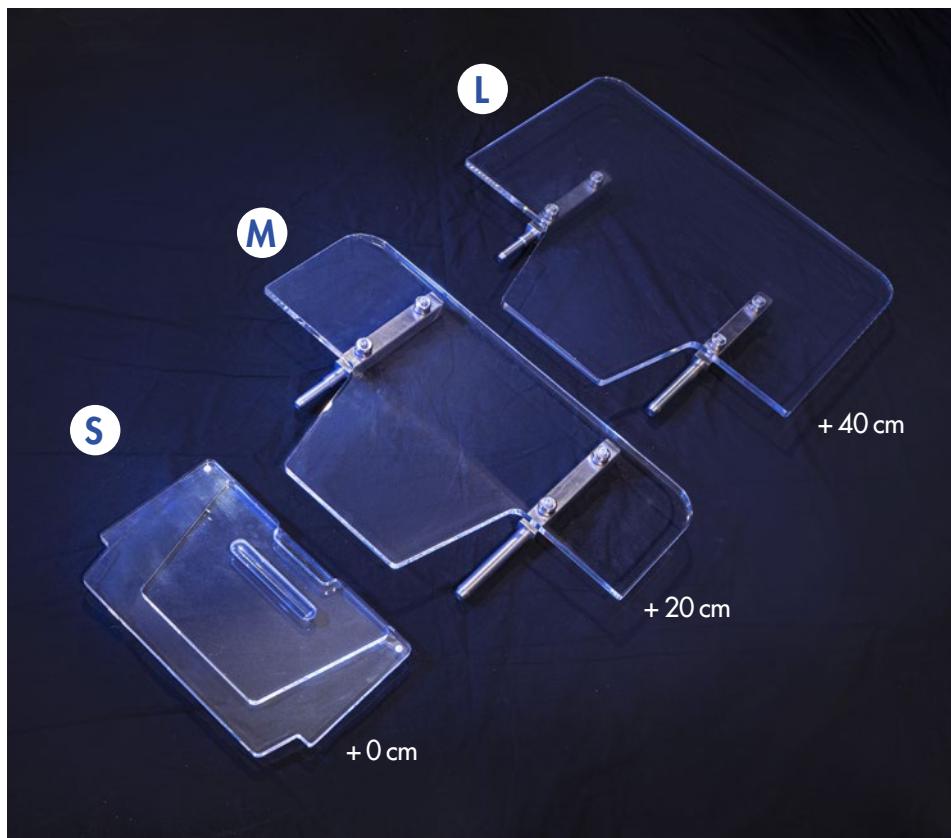
The small extension (tabletop + 0 cm) closes the notch of the main tabletop, reducing the table's space for procedures requiring minimal equipment (carpal tunnel release, trigger finger procedures...).

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The medium extension (tabletop + 20 cm) enables the performance of most standard surgical procedures on the upper limb.

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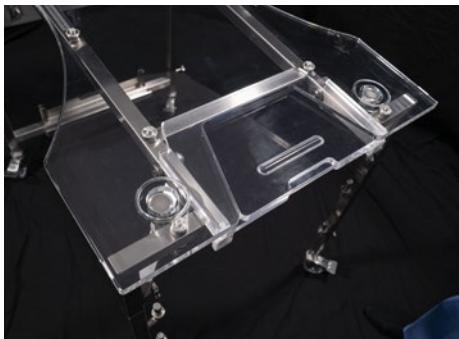
The large extension (tabletop + 40 cm) provides ample space during certain ancillary procedures (osteosynthesis, prosthesis...).

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#### ASSEMBLY:

Once mounted, the small extension fills the notch in the main tabletop unit.

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#### STORAGE:

The small extension is stored in its dedicated slot on the crossbar.

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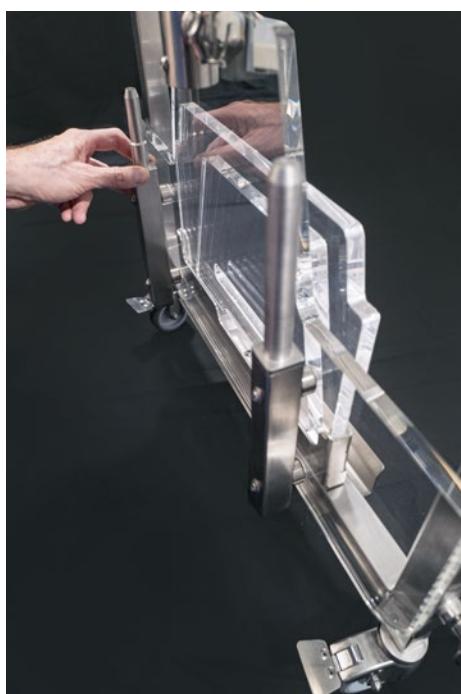
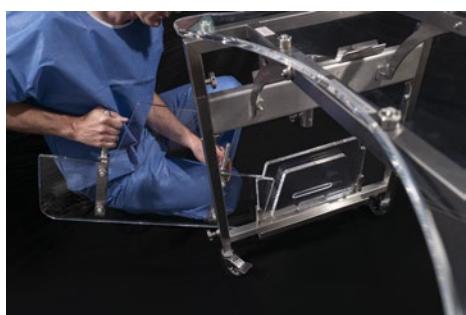
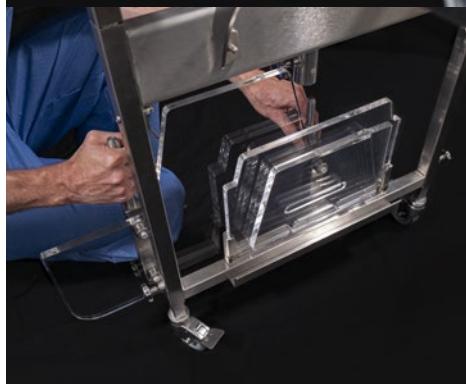
#### STORAGE:

The medium extension slides into the rail on the outside of the table's crossbar.

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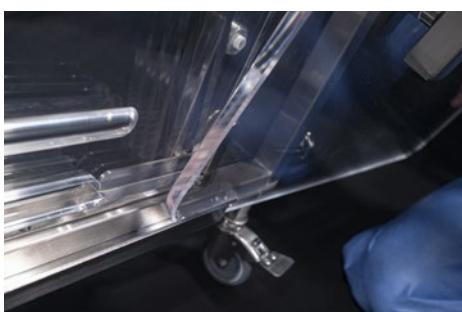
#### ASSEMBLY:

The large extension fits by inserting the metal rods into the white sockets on the table's edge.

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#### STORAGE:

The medium extension slides into the rail on the inside of the table's crossbar.

The hinged hooks placed on both sides of the frame hold the extension in its storage.

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
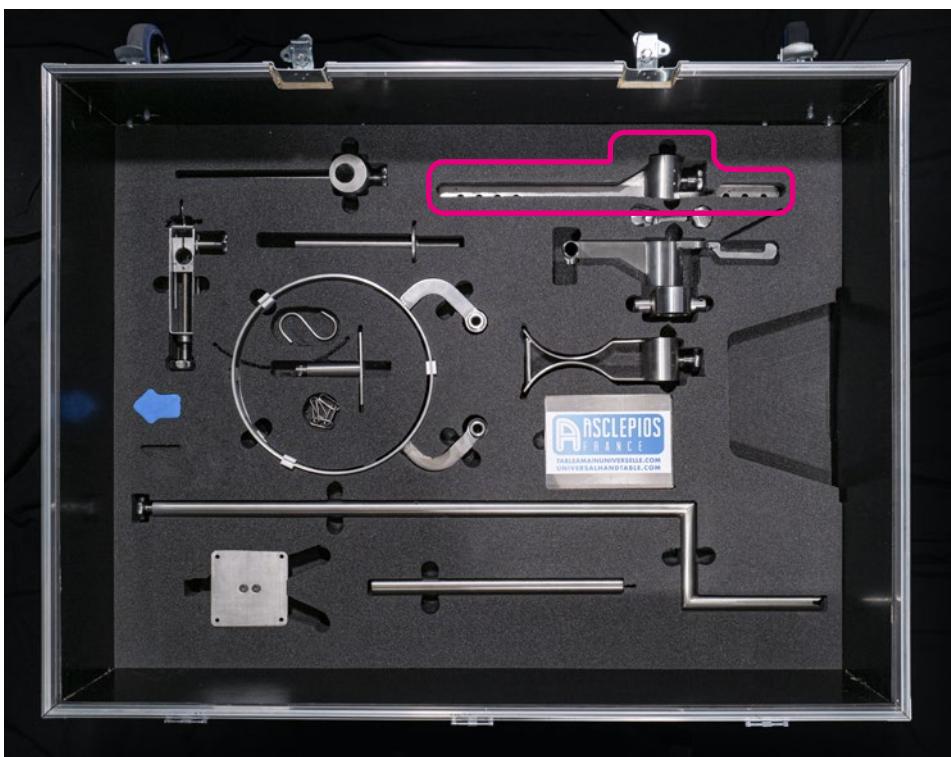
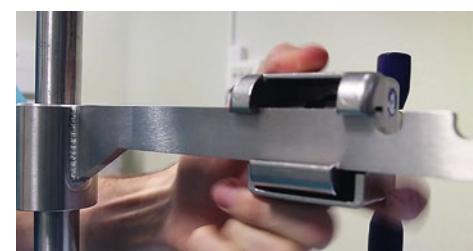
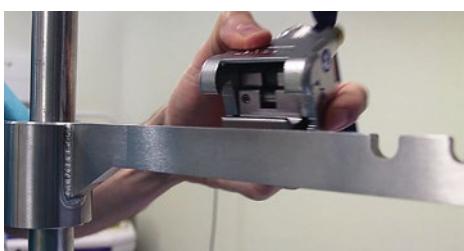
Designed to hold the surgical drape, its cross-section matches the rails of standard operating tables, allowing it to accommodate any clamps designed for those rails.

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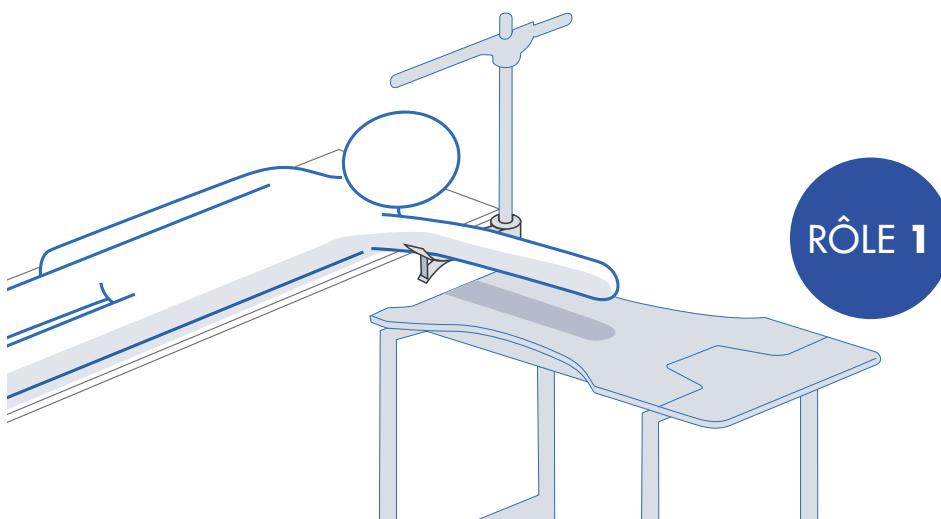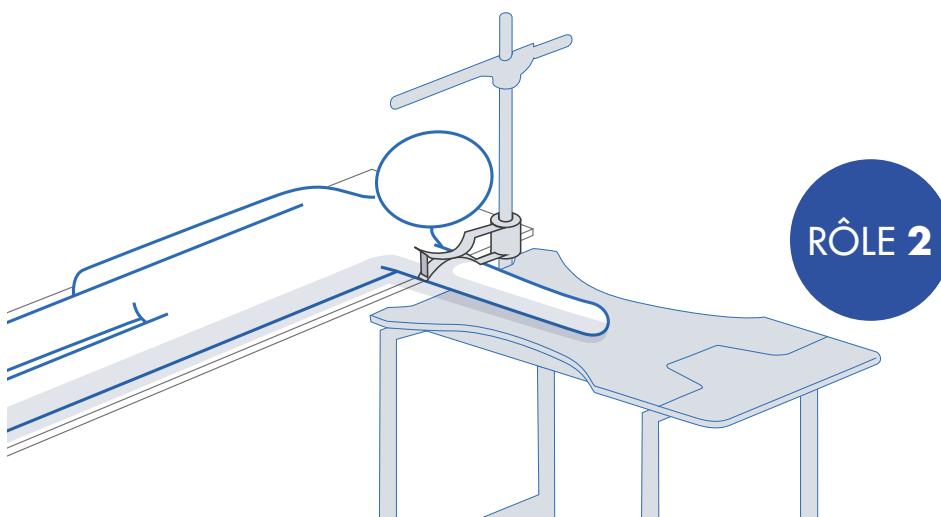
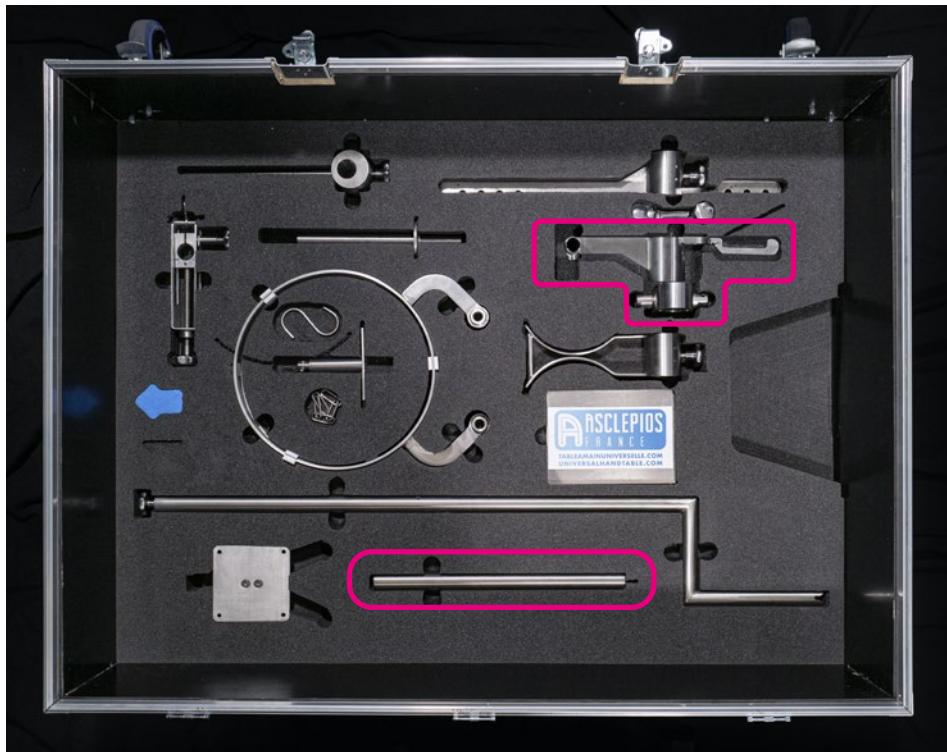
Extension de potence / Support arm extension

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This is a prerequisite for wrist arthroscopy, as it extends the support arm and provides an anchoring point for traction.

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
Assembly and storage :  
see page 43

## OPTION 7 :

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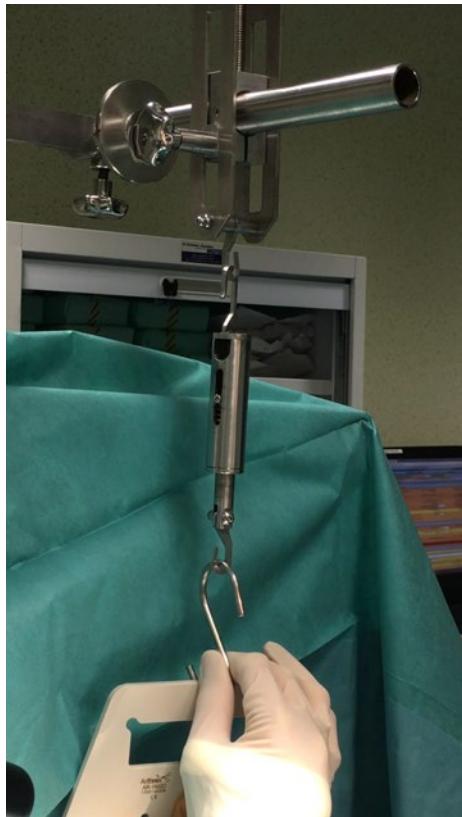## Arthroscopy ancillary instruments

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
The arthroscopic ancillary instruments are reusable  
sterilisable accessories dedicated to wrist arthroscopies.

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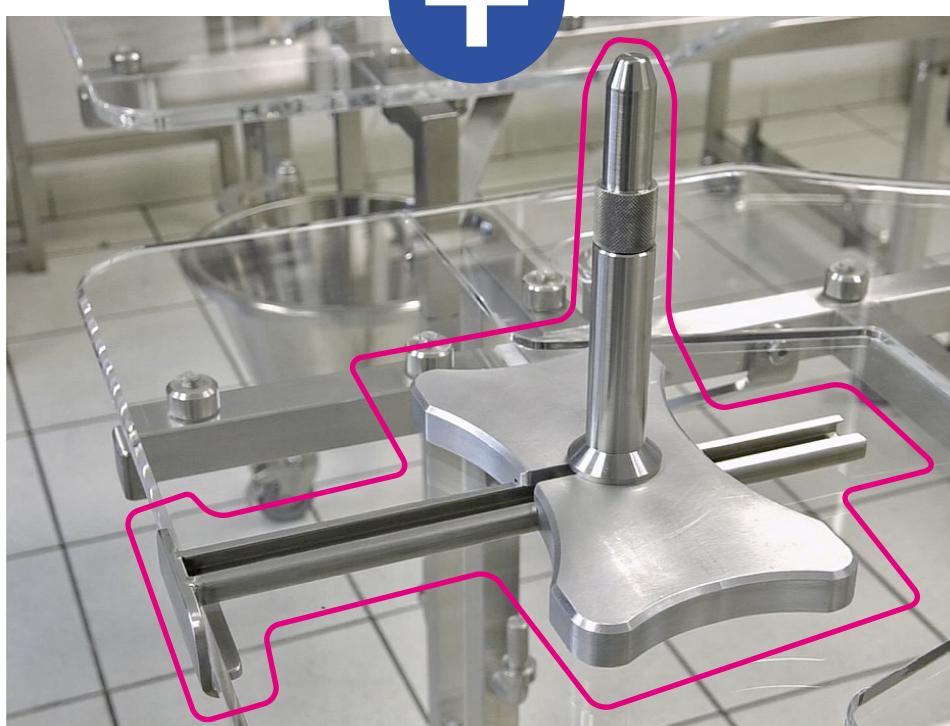The horizontal traction kit may prove useful for wrist-fracture reduction. Please note that OPTION 7 (arthroscopy ancillary instruments) is required to use this option.

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The “Alsatian” traction method stems from the ingenuity of the team at Strasbourg University Hospital. In their routine wrist and hand arthroscopy work, they found it challenging to generate a strong traction force along the forearm's axis.

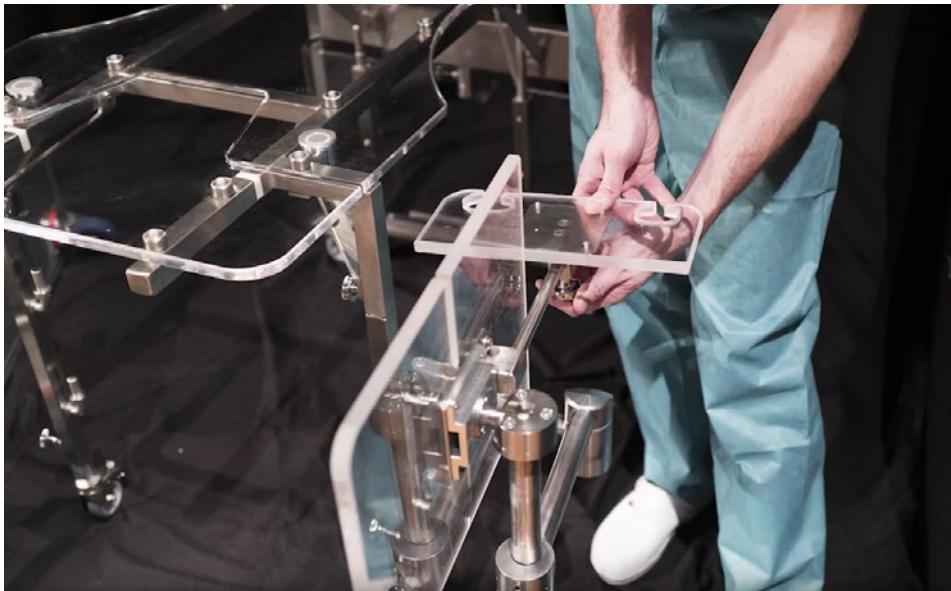
They therefore devised a vertical traction aid featuring a gear-reduced rack-and-pinion system. However, it is important to be aware that this system is extremely powerful and must be used with the utmost caution.

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The Mayo stand consists of a set of modular components.

It can, for example, serve as a resting place for instruments when you need to remove the small extension to slide the image intensifier into its dedicated slot.

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Some surgeons wish to position a tablet, mirror or display on articulated support arm offset from the device's main axis.

For this specific application, a dedicated system accommodates the shaft of an articulated arm, and the assembly then locks onto the support arm at an adjustable height.

## OPTION 12 : Support éclairage Lighting support

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A genuine operating-room light isn't always essential for surgery—and of course even less so for consultations.

A lighting module is available to illuminate the table surface. As with all the other options, its support can be adjusted to the desired height.

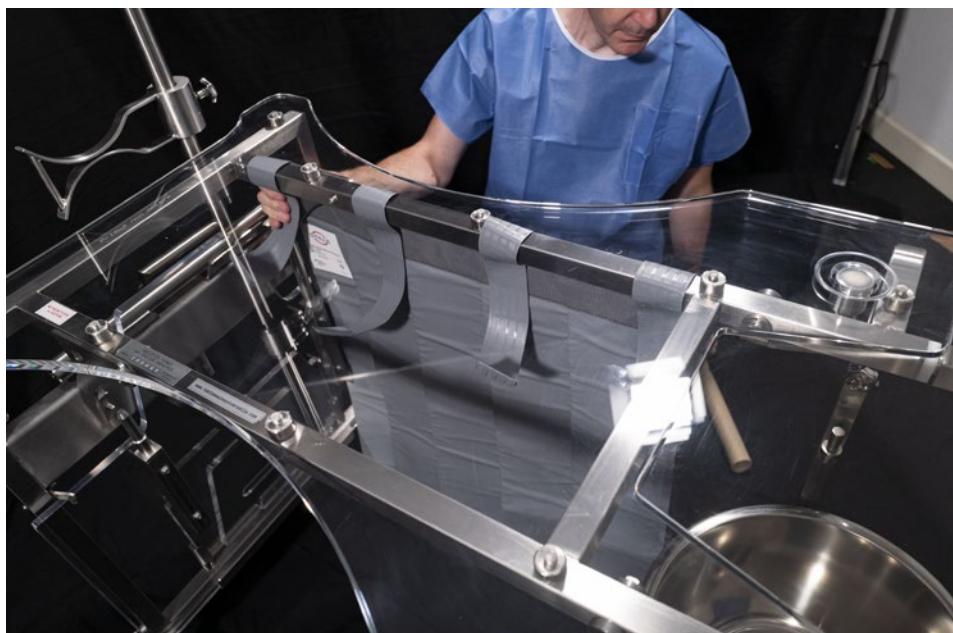
Energy-efficient, this LED spotlight can run on battery power or be plugged directly into mains.

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These lower X-ray shutter panels serve as an additional radiation-protection measure when using an X-ray generator.

Unless specific circumstances dictate otherwise, the X-ray tube should be positioned high and the receptor placed close to the floor. Only the operated patient's hand should be in the primary X-ray beam. Of course, fluoroscopy time must be kept as short as possible.

Most scattered X-rays return at torso level to the surgeon and assistant, so both **must be protected by lead aprons.**

These precautions, combined with the use of a mini C-arm, significantly reduce exposure for the medical staff. Nevertheless, a small proportion of the emitted rays will inevitably be directed downward toward the lower parts of the surgeon's and operating assistant's bodies.

The use of protective lower X-ray shutter panels **offers limited additional shielding** for the only area still exposed—the lower legs and feet.

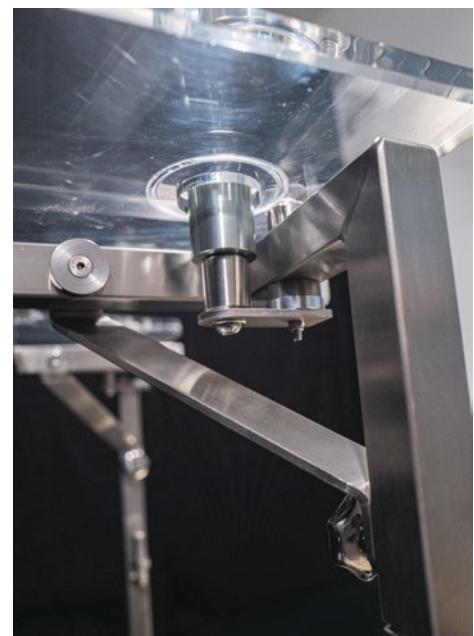
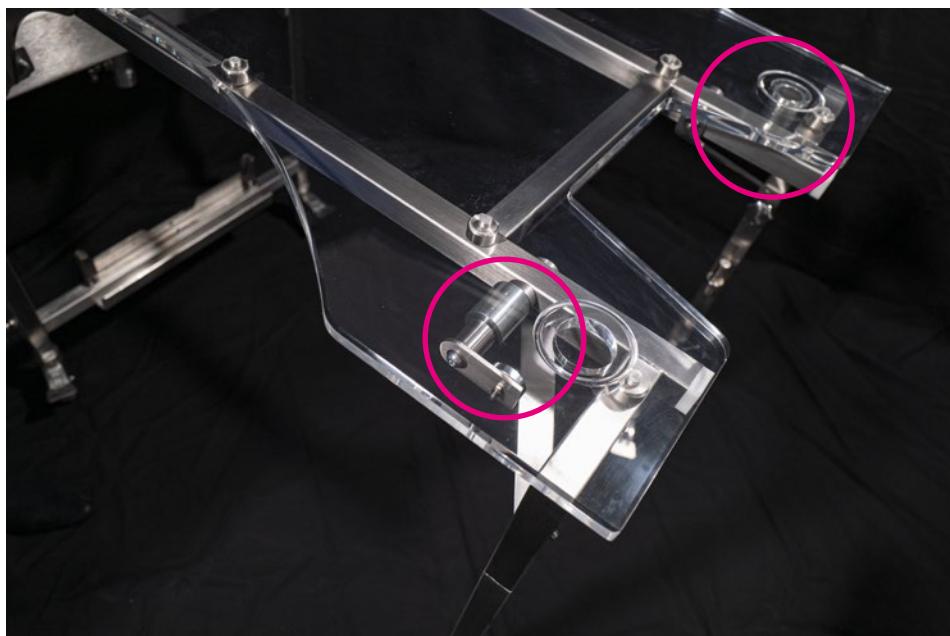
However, it is important to note that these lower panels do not replace the aforementioned optimisation measures for reducing exposure to ionising radiation.

The shutter panels are available in blue or grey.

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The two magnets mounted beneath the tabletop attract sharp or pointed objects.

When not in use, they can be removed and stowed on their dedicated holders beneath the tabletop.

These removable magnets can also be used to locate and retrieve a needle dropped onto the floor.

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The dedicated Asclepios stool provides both lumbar support and excellent mobility.

The backrest's dimensions and position have been designed to avoid interfering with the arms and elbows.

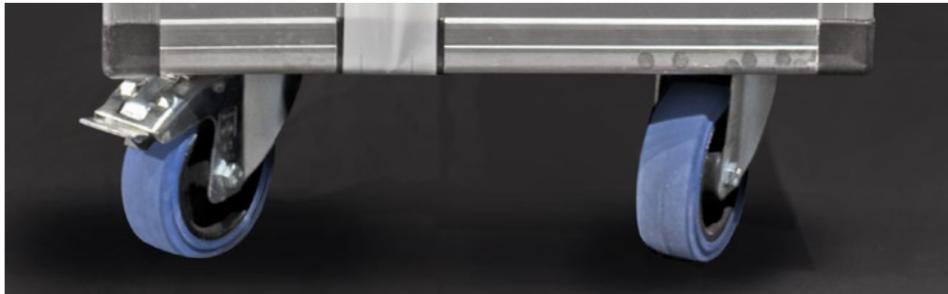
A foot pedal allows you to adjust the stool's height while keeping your hands free.

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The carrying case moves easily on its four swivel wheels. It is equipped with handles.

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
Only one side of the case is fitted with stops (circled in pink in the photo). **This is the side on which the case must be laid horizontally to access its contents.**

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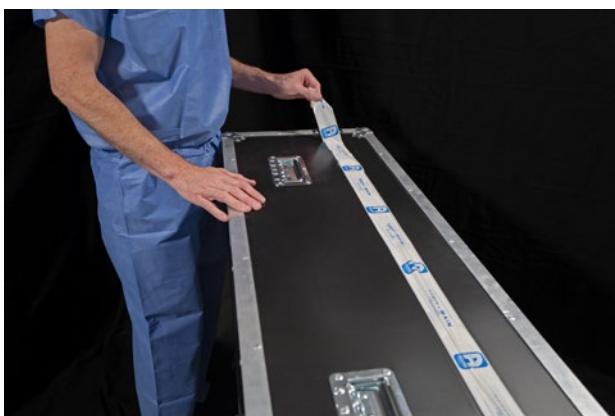**Warning: Never attempt to open the case while it is resting on its wheels.**

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With assistance from a colleague, tilt the case into a horizontal position using the handles. Caution: when tipping, expect a slight shift as the swivel wheels rotate.

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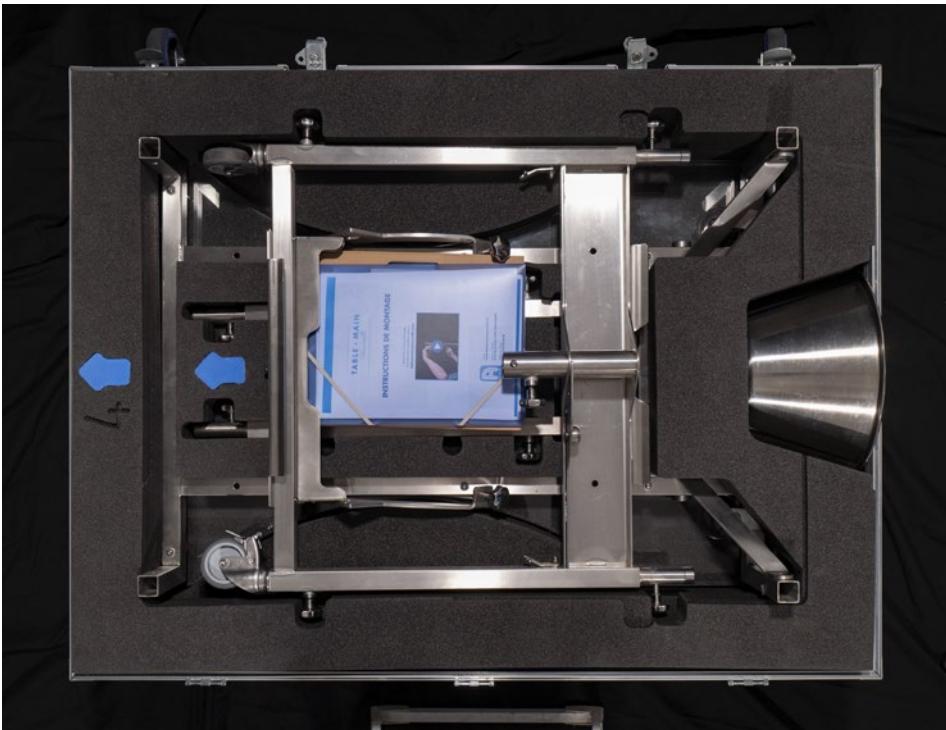
Once the case is laid flat on the floor, it can be opened by unlocking the butterfly latches on the wheel side.

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
Remove the case lid by detaching its removable hinges.

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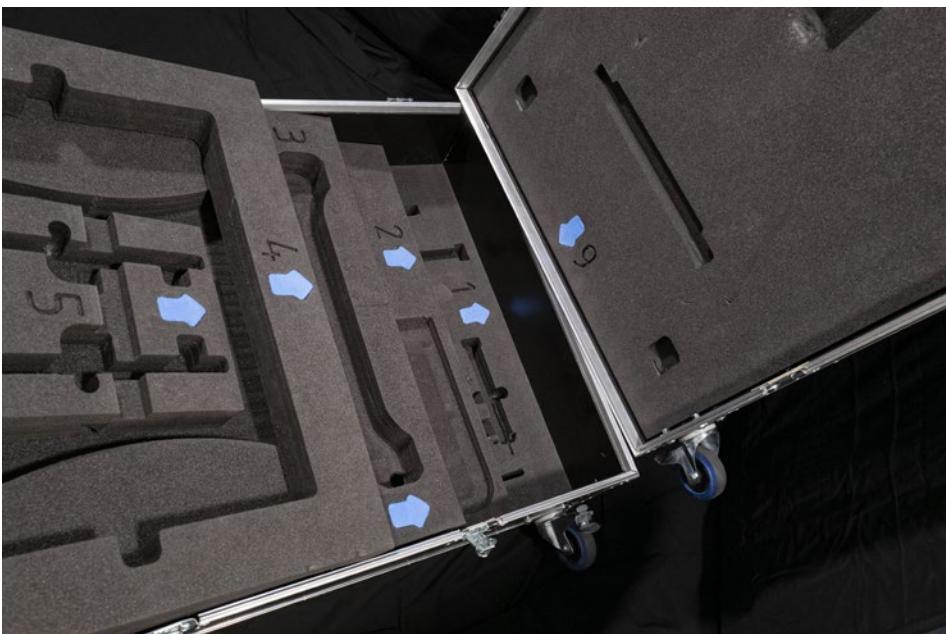The storage inside the case is organized in layers of protective foam.

Each foam layer is numbered from 1 to 6 to help you restore the correct stacking order when packing the table. Start with layer 1 at the bottom of the case, then proceed layer by layer up to layer 6, which fits into the lid.

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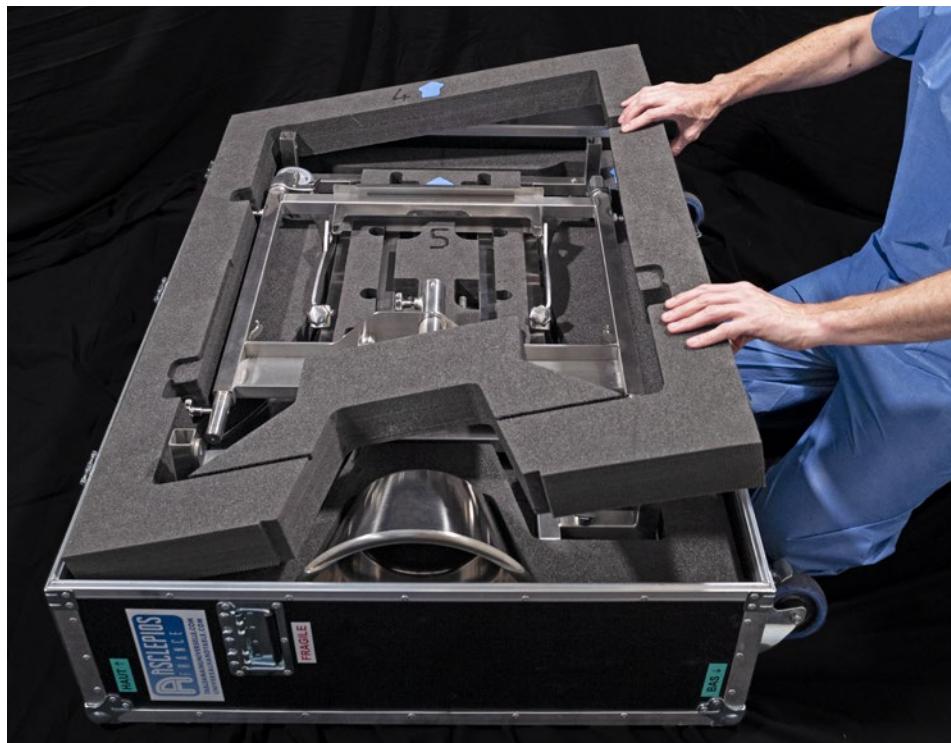
Assembly requires no tools; all table accessories are contained in the transport case.

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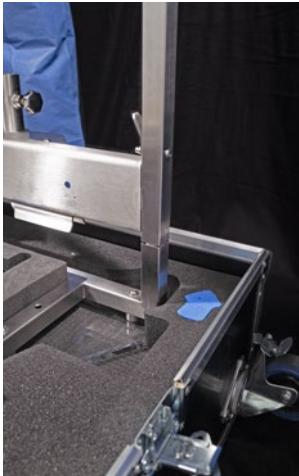
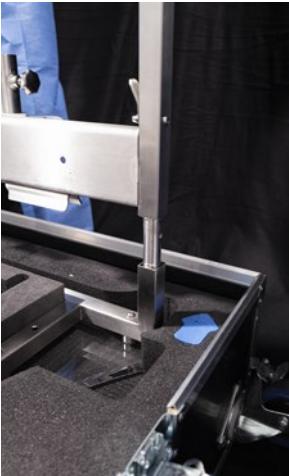
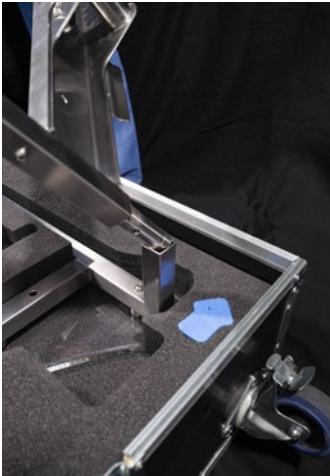
Remove and set aside the two screws on either side of the wheeled frame on the blue-marked side, opposite the bin.

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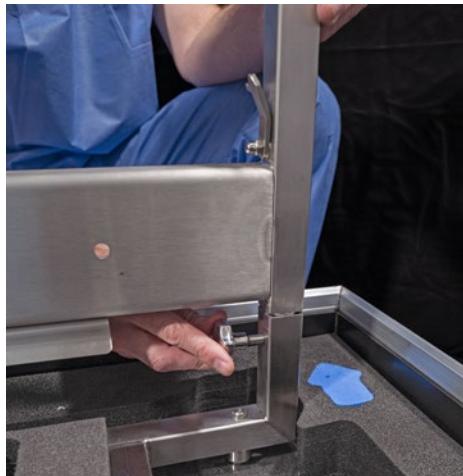
Take the wheeled frame and slide it onto the two posts provided for this purpose, always on the side opposite the bin.

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Screw the two screws that were initially removed to secure the wheeled frame to the tabletop.

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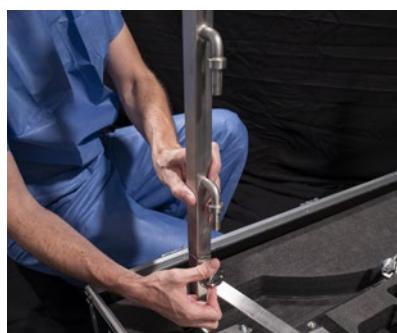
Take out the first of the two remaining caster legs.

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
Slide it on as before, ensuring that the two hooks (which will secure the trash bins) face inward. Then screw the leg into place.

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
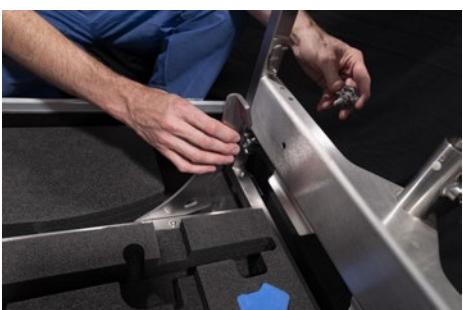
Repeat the same operation with the last leg.

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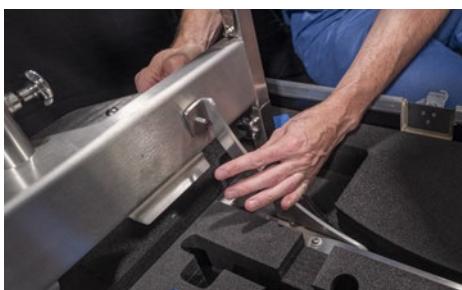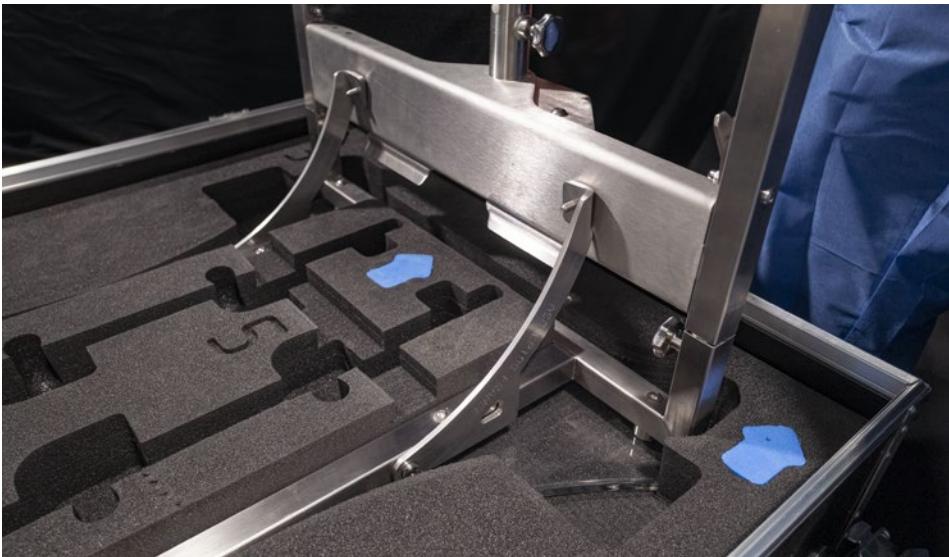Lift up and screw in the two curved reinforcements on the frame side.

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**Manually check  
that all screws are  
properly tightened  
before continuing the  
assembly.**

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Adjust the table height to your preference, one leg at a time. When the legs are fully retracted, the tabletop height is set at 80 cm. **Each notch raises the tabletop by 2.5 cm**, up to a maximum of 105 cm.

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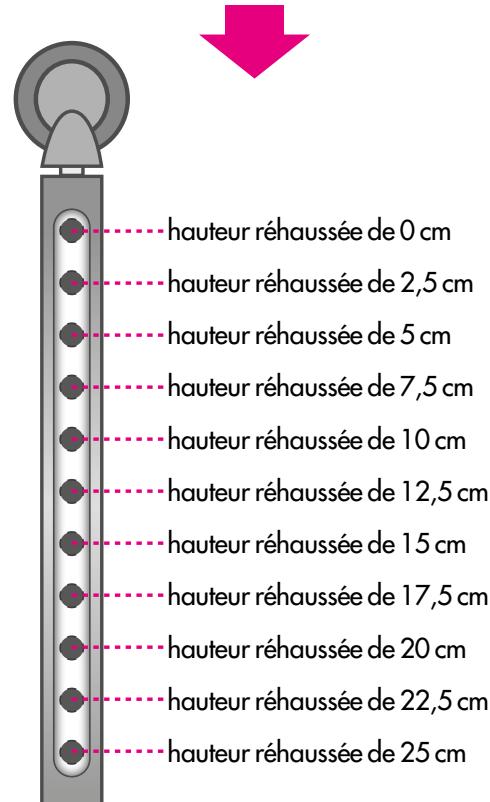
Adjust the three legs to the required height, ensuring they are all set to the same level.

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
The fourth leg has no notch. It will only be adjusted later, once the table is resting on its legs, to compensate for any unevenness in the floor.

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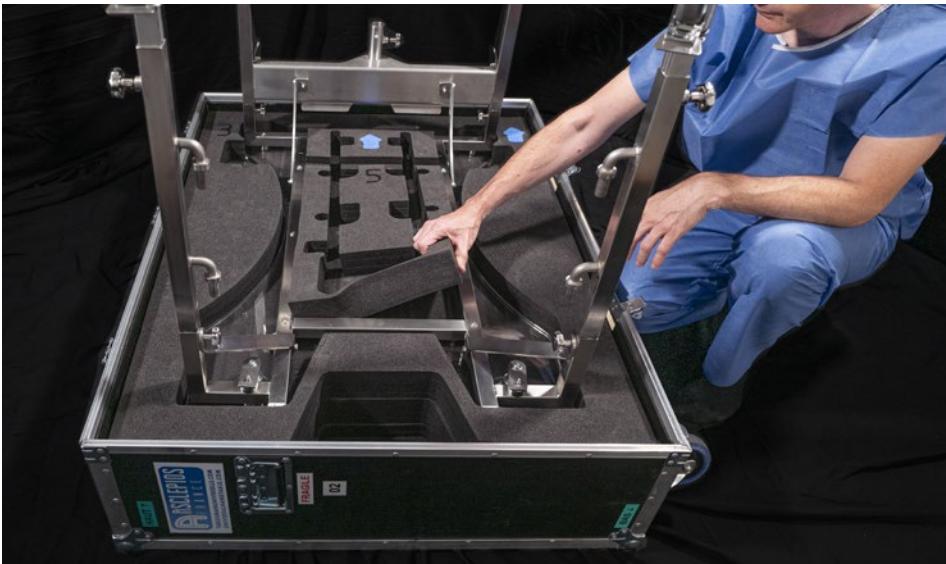Remove and set aside the two removable bins (they are nested one inside the other).

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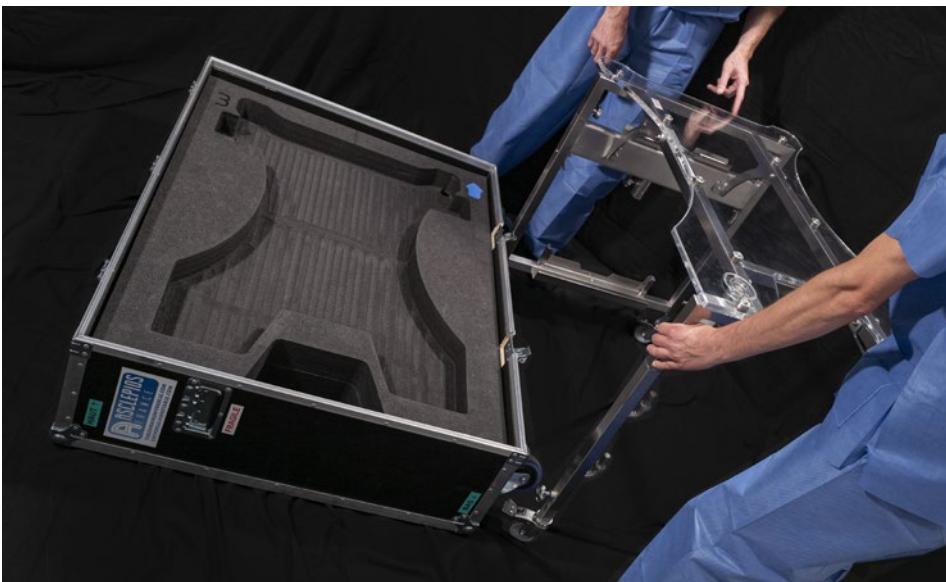
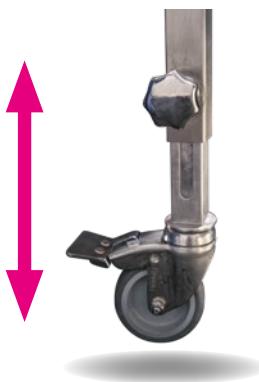
Level your table using the adjustable foot. Unlock the screw and allow the wheel to touch the ground. Then tighten the screw.

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Lock the table using the wheel brakes.

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
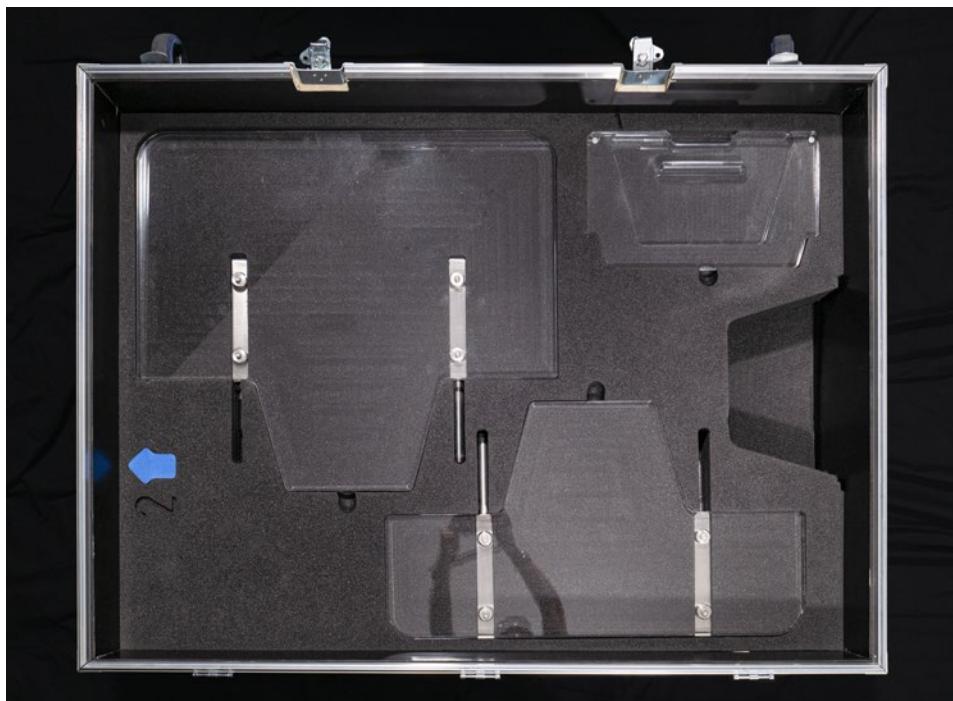
Remove protective foam piece No. 3 to access the rest of the contents of the case.

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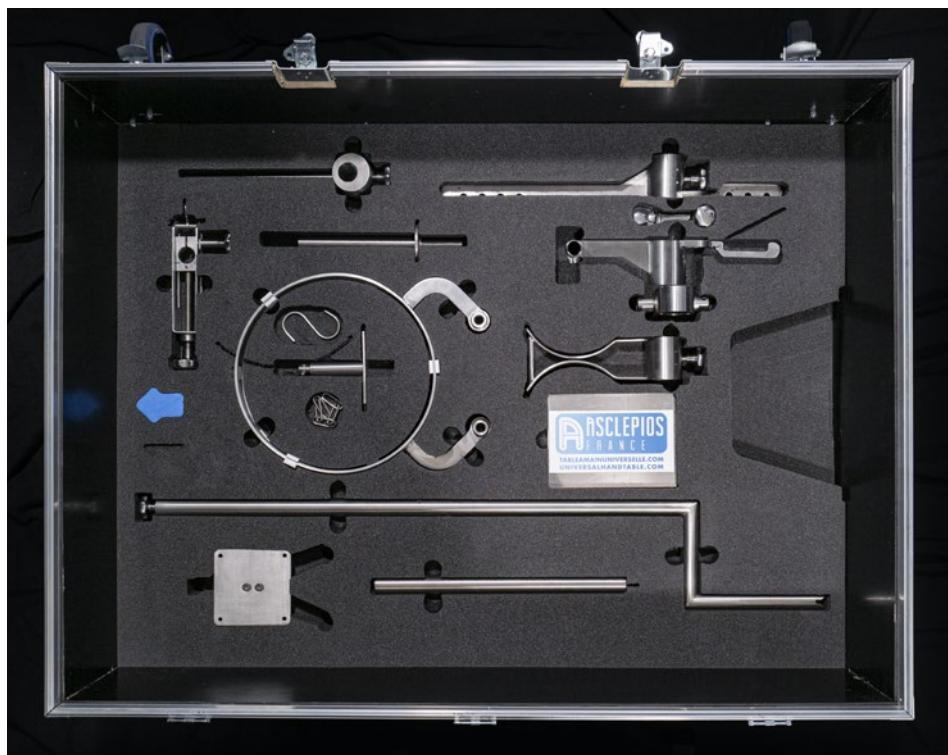## **Step 4: Installation of the support arm and its accessories for vertical traction**

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Remove protective foam piece No. 2 to access the various accessories in the bottom layer.

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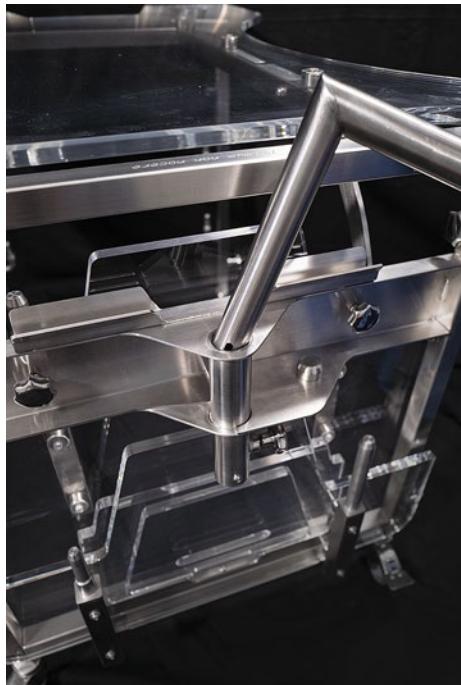
First, take the support arm.

Remove and set aside the screw at its end.

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Slide the support arm onto its shaft. It can be mounted in either of two preset positions: right or left.

## ÉTAPE 5 :

### Support de bras Step 5: Arm support

Prenez le support de bras.

Take the arm support.

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Slip it on from the top of the support arm.

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**Slide-on parts are not symmetrical. They have a “lower” side and an “upper” side (see photos on the left) and must be fitted via the lower side by presenting the part at an angle.**

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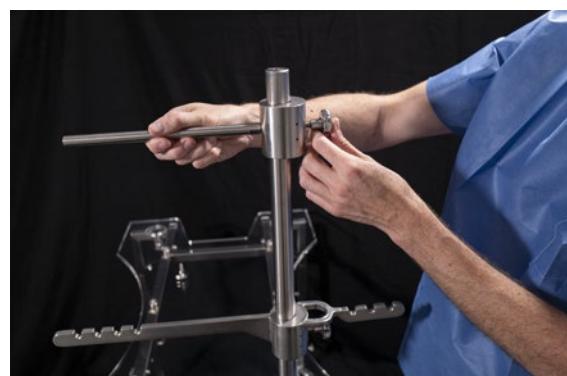
Slip it on in the same way as the arm support.

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Take the support shaft intended for the mirror/tablet holder.

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Slide it on in the same way as the arm support.

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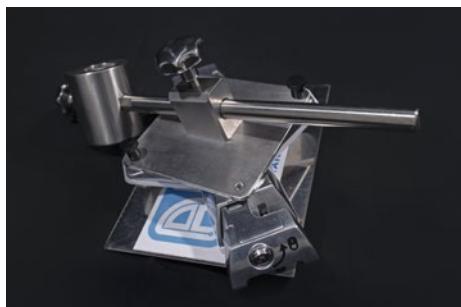
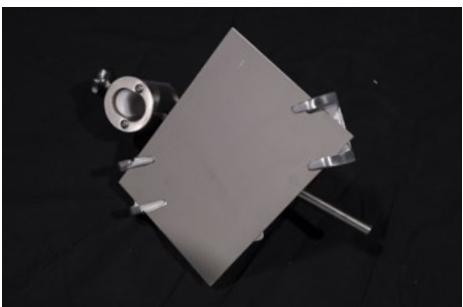
Finally, take the mirror/tablet holder and the mirror.

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The mirror/tablet holder uses a claw mechanism that opens and closes with a locking system.

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**Note that this piece is not symmetrical: it must be installed with the angled side facing the pole for mirror use, and oriented the opposite way for tablet use.**

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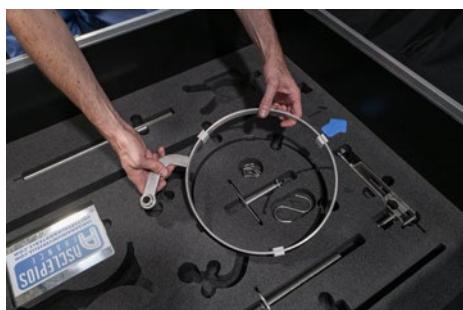
Finally, if you are not using the arthroscopy accessories, replace the screw at the top of the support arm. It serves as a spare should any other screw go missing, and its larger diameter prevents any support from accidentally slipping off the support arm.

## ÉTAPE 6 :


### Montage des derniers accessoires

### Step 6: Fitting the remaining accessories

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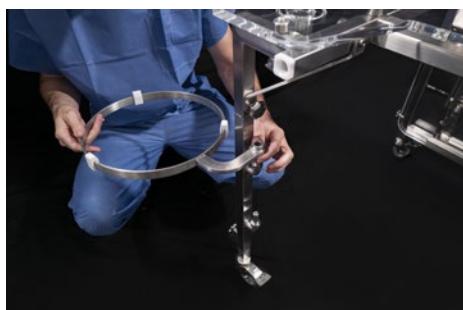Slide them onto the table legs on either side.

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Place the bins in their support hoops.

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## Step 7: Assembly of arthroscopy accessories

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Take the support arm and the arthroscopy support bracket.

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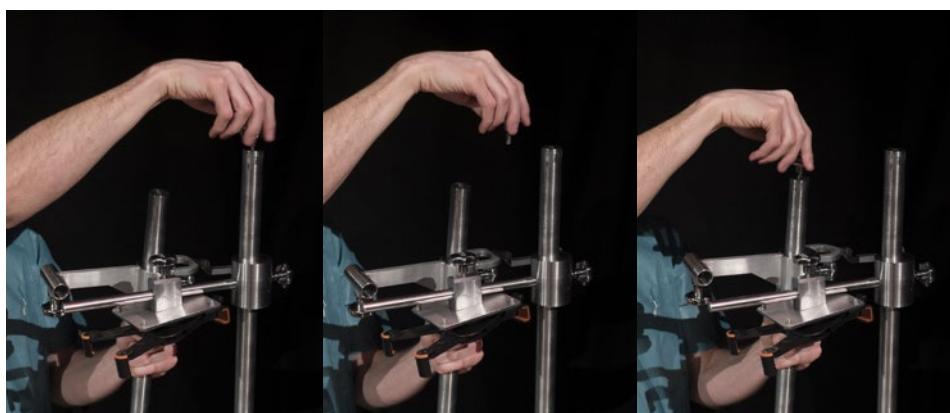
Insert the support arm extension into the arthroscopy support bracket.

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
The arthroscopy support is also asymmetrical. It has a "lower" side and an "upper" side (see photo on the left and the "arm support" page, where the same orientation principle applies), and the support arm extension must be inserted through the lower side by slightly tilting the piece.

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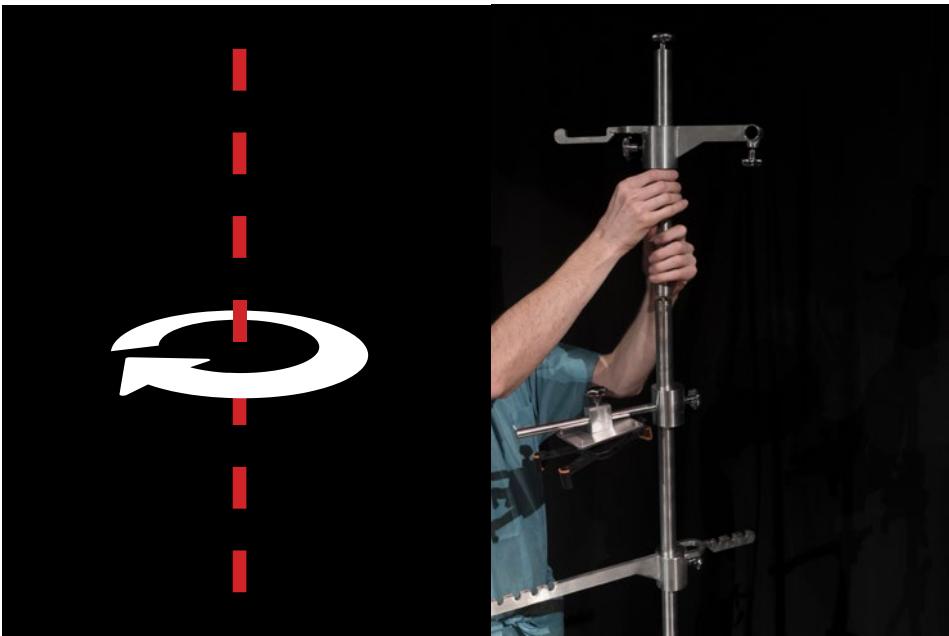Now remove the screw from the top of the main pole and screw it into the support arm extension, as doing so later would be difficult due to the height.

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
Screw the support arm extension, equipped with its support, onto the main pole. Be mindful of the arthroscopy support's rotational clearance during tightening.

Ensure the screw is fully tightened — both parts must be fully interlocked.

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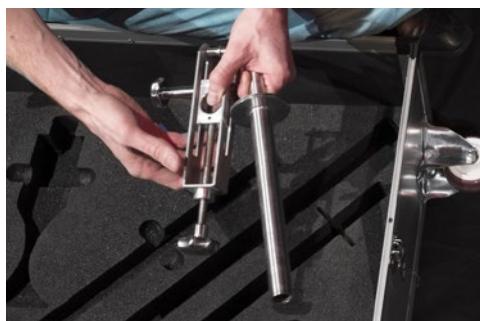The first set of equipment can be operated under sterile or non-sterile conditions, as chosen by the operator. The order is identical in both scenarios:

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**Tighten the tensioner's side screw securely.**

**CAUTION: Never tighten this screw empty (i.e., without inserting the support rod first), as you will damage the tensioner!**

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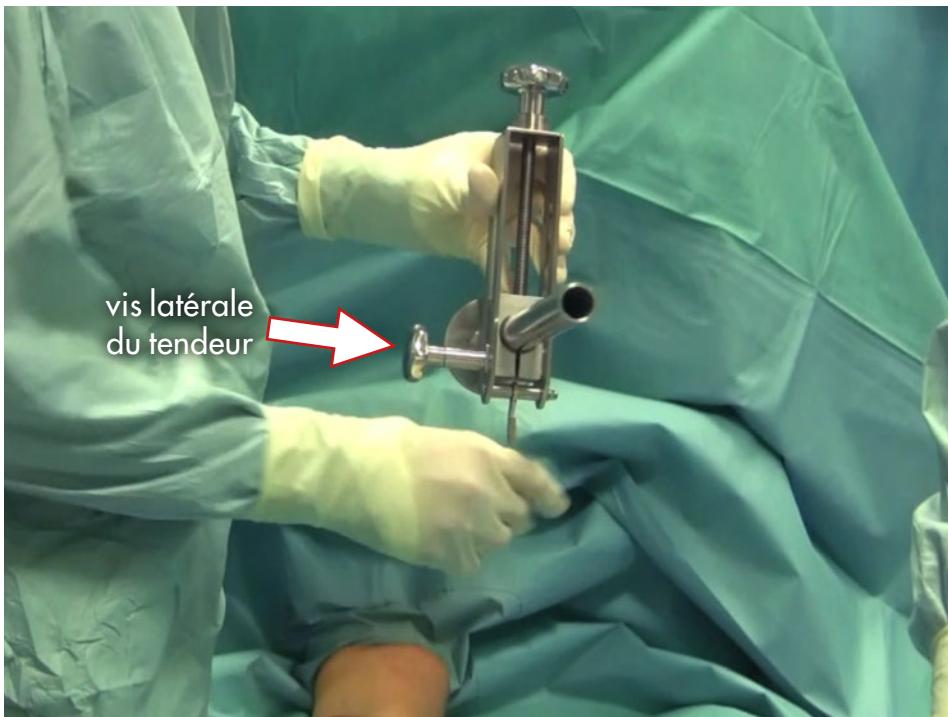
Insert the support rod/tensioner assembly into the arthroscopy support.

**! Serrez bien la vis du support d'arthroscopie.  
Firmly tighten the screw on the arthroscopy support.**

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### 1.2 - Use of the ancillary in fully sterile conditions:

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The first set of accessories may be used either in sterile or non-sterile conditions, as chosen by the operator. In both cases, the sequence remains the same:

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Insert the support rod (sword-shaped rod) into the tensioner.

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**Tighten the tensioner's side screw securely.**

**CAUTION: Never tighten this screw empty** (i.e., without inserting the support rod first), **as you will damage the tensioner!**

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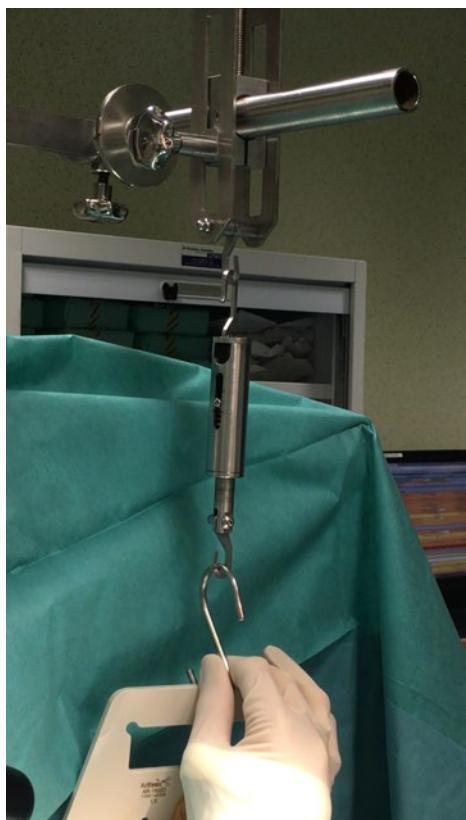
**Firmly tighten the screw on the arthroscopy support.**

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The spring scale must be pre-equipped with its hook. It may be used under sterile or non-sterile conditions.

The hook is reversible. However, Hook **B** can attach directly to the Finochietto handle.

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The stainless steel S-hook must be pre-sterilized (for example, in double sterile pouches). The pouches are opened.

The S-hook serves as the interface between the spring scale (sterile or non-sterile) and the Finochietto handle (sterile).

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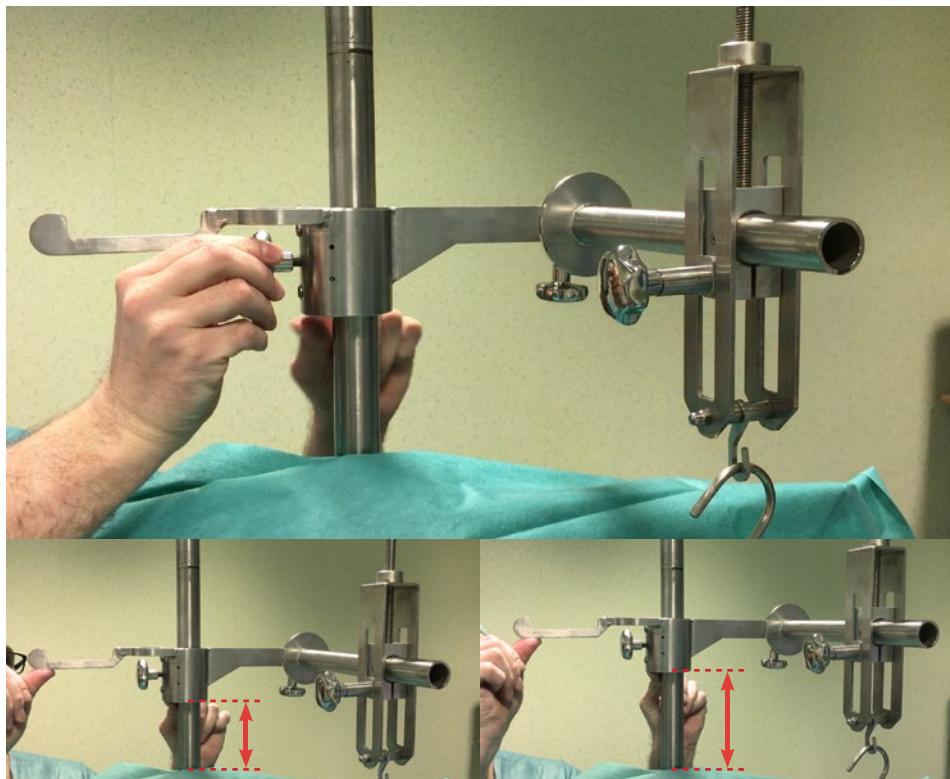
The surgical spring scale, already equipped with its rack-style (forked) end, must have been sterilised before use (e.g., in double sterile packs). While the stainless steel S-hook is optional, it is recommended that it is also sterilised before use.

Pass the spring scale hook into the tensioner hook (or through the S-hook), then attach the sterile Japanese finger traps to the forked end.

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Those new to the universal hand table frequently assume the tensioner provides the traction force. In reality, almost all of the traction is generated by raising the arthroscopy support.

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Under non-sterile conditions, this step is performed by the operating room nurse. The surgeon monitors the traction force as it increases by reading the graduations engraved on the surgical spring scale.

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This rigid, deep-angled hook is positioned toward the patient's hand and accommodates the Finochietto handle.

It is important to note that the two hooks at each end are not identical...

This rounded, movable hook is positioned on the tensioner side; it accommodates the stainless steel S-hook or the rings of the stainless steel traction chain.

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The tensioner's long adjustment screw serves solely to fine-tune the traction, making it a little tighter or a little looser. Therefore, before traction is generated, the tensioner carriage should be placed at an intermediate setting in order to work in both directions.

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The marks on the spring scale indicate 3, 6, and 9 kilograms, respectively.

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If it proves challenging for the personnel to produce traction, the surgeon can apply it directly through the sterile chain.

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
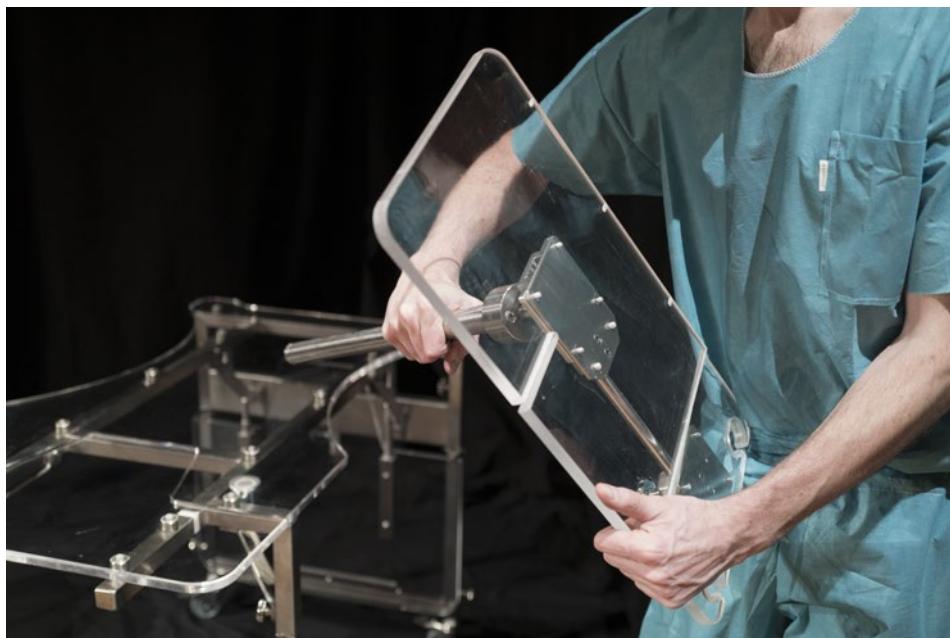
Take the articulated shaft and place it on the selected table leg.

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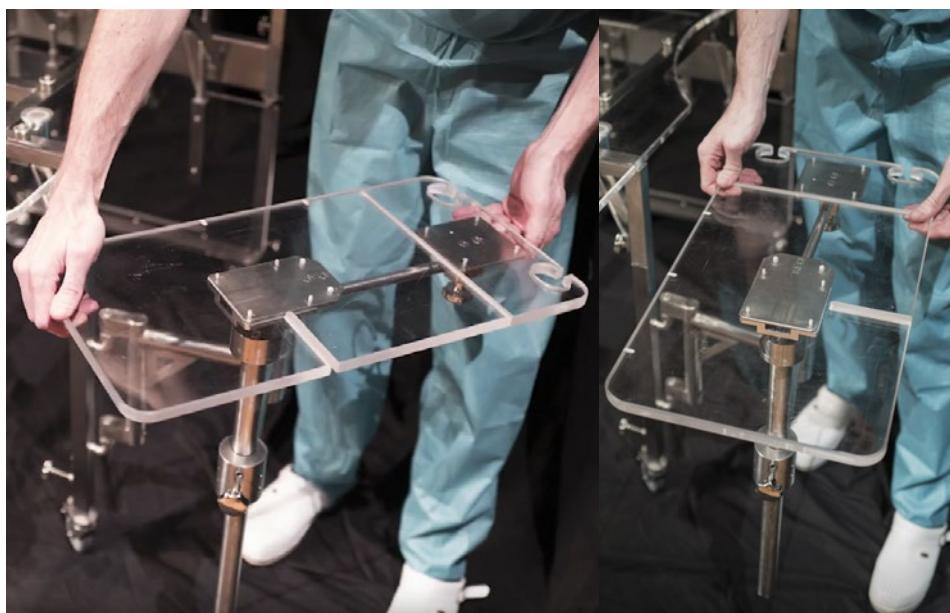Swing the shaft open so it is perpendicular to the clear plate.

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
Engage the plate into the support shaft.

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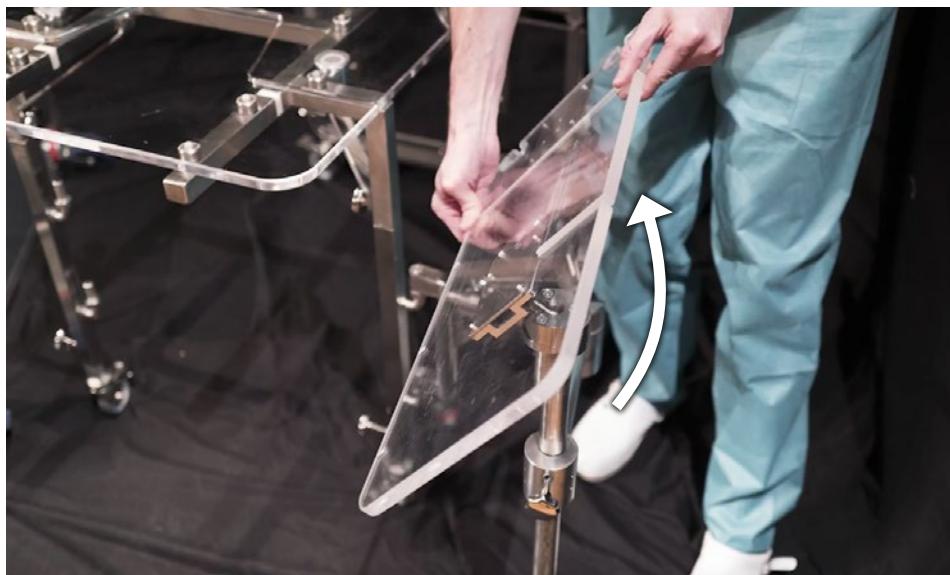
The table can tilt to a vertical position for tablet-style ultrasound systems.

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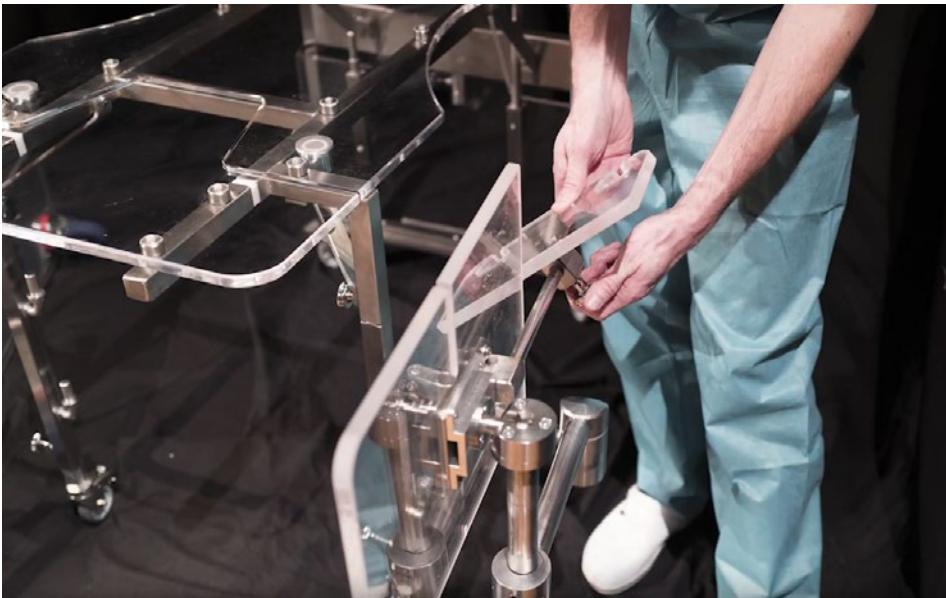
The probe support can be oriented independently of the main tabletop unit...

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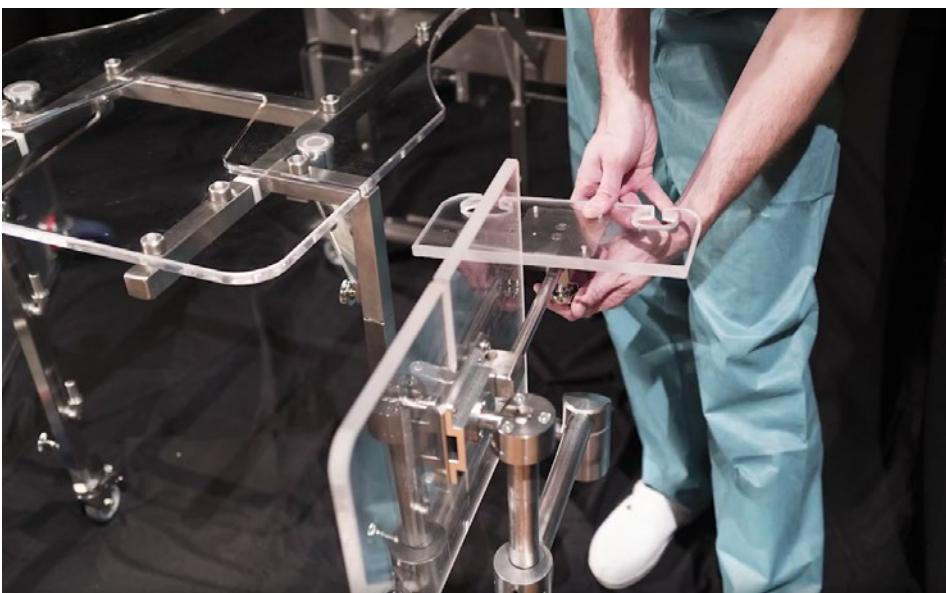
Remember to secure the lock after selecting the desired orientation.

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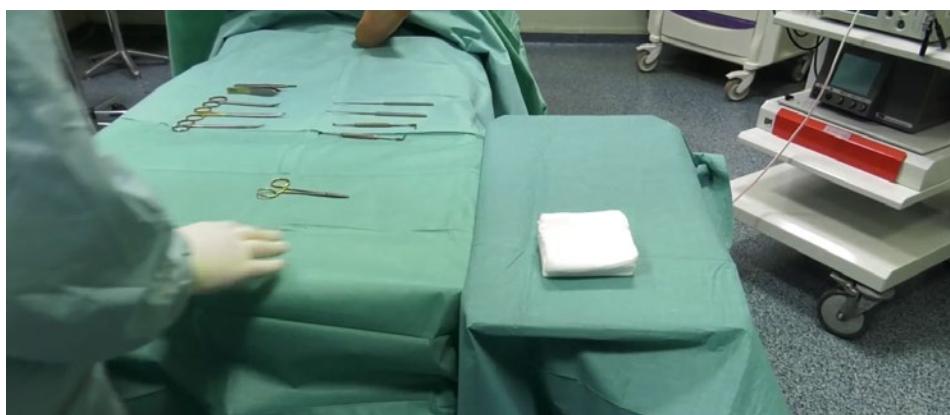## **Using the Mayo stand in sterile conditions**

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Primarily for foot procedures under scope.

Flat set-up use.

Provides a holding area for instruments when the small-extension is pulled off to move the image intensifier into its reserved notch.

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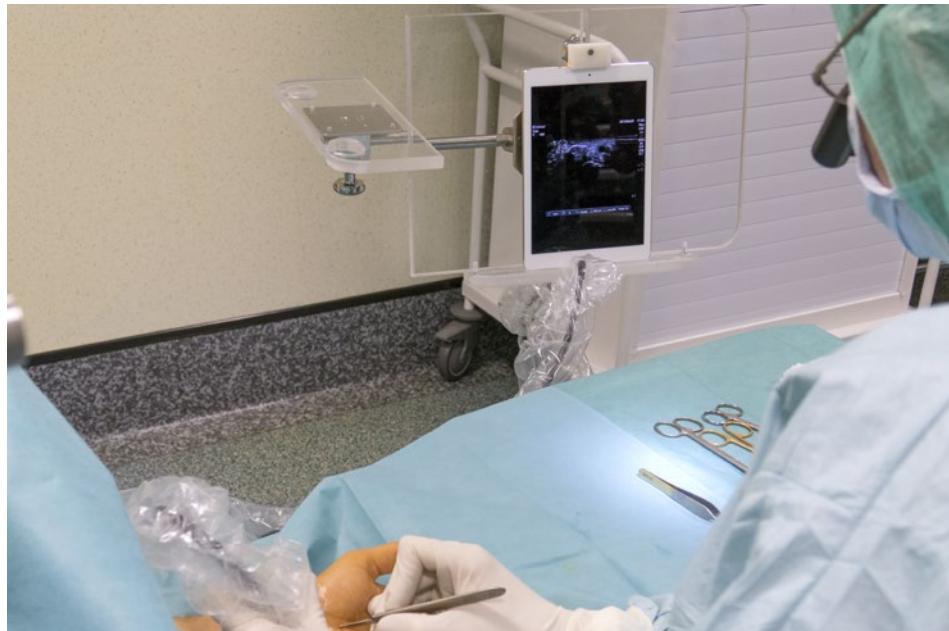
## **Use of the Mayo stand in NON-sterile conditions**

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
Use with a vertical ultrasound unit.

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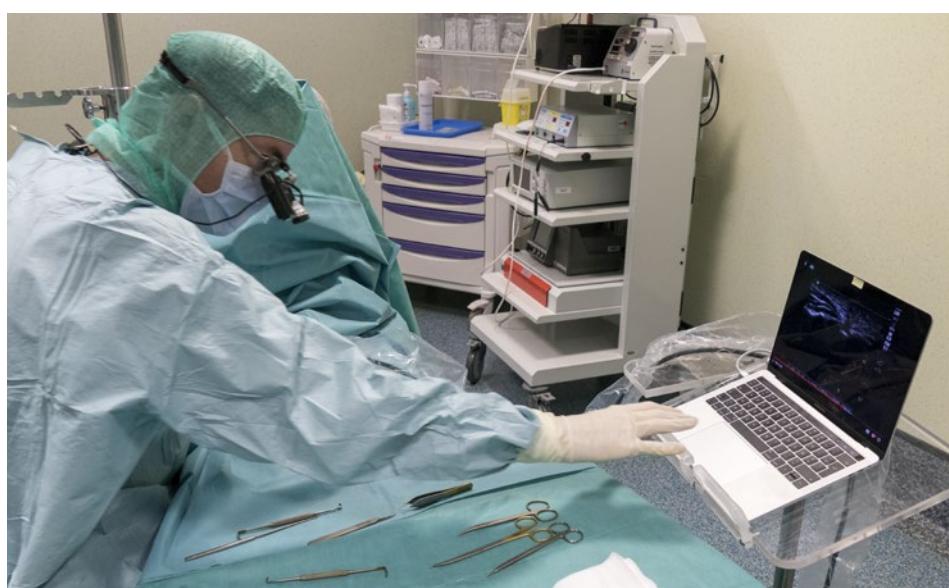Use with a tablet-based ultrasound unit.

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Intended for use with a laptop ultrasound system (to be used with a sterile keyboard protection- not provided).

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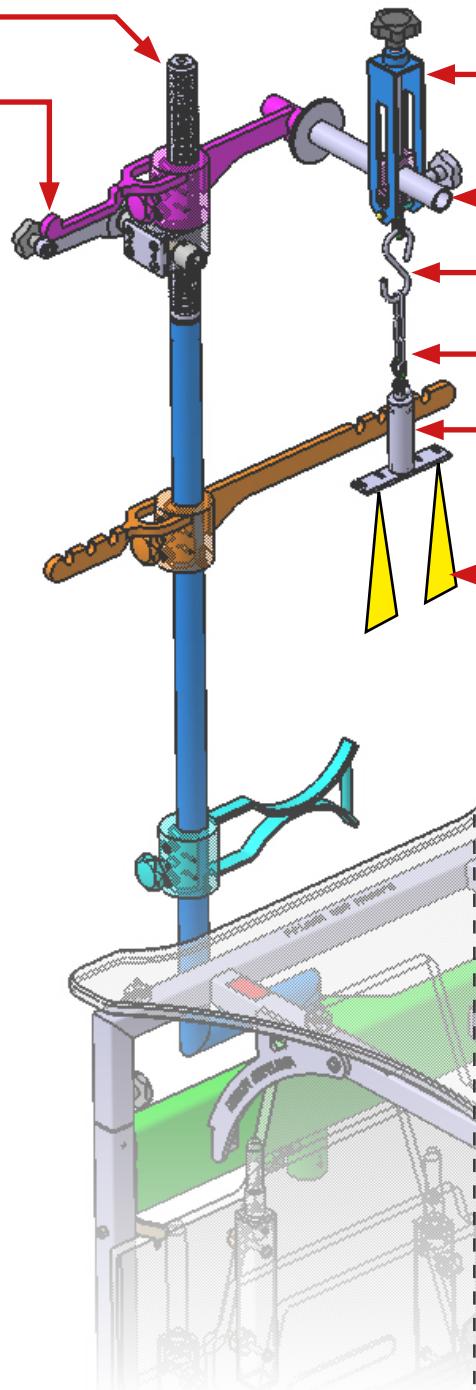
## Appendix 2 : Description of the parts/accessories for use with Japanese finger traps

Called "EXTENSION AND ARTHROSCOPY HOLDER", one unit per non-sterile table is sufficient; it does not require sterilisation.

Support arm extension

Traction holder for performing arthroscopy

Called "ARTHROSCOPY SET," one unit per arthroscopy operation; it is sterilised between uses.

Tensioner: enables regulation of the traction force

Support rod: this is the support shaft; it fits into the arthroscopy support

An optional S-shaped hook; allows for even faster installation

Traction chain: five rings (optional use) to assist in establishing the initial traction

Spring scale: indicates the traction weight applied by the system, adjustable via three settings (3, 6, or 9 kg)

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
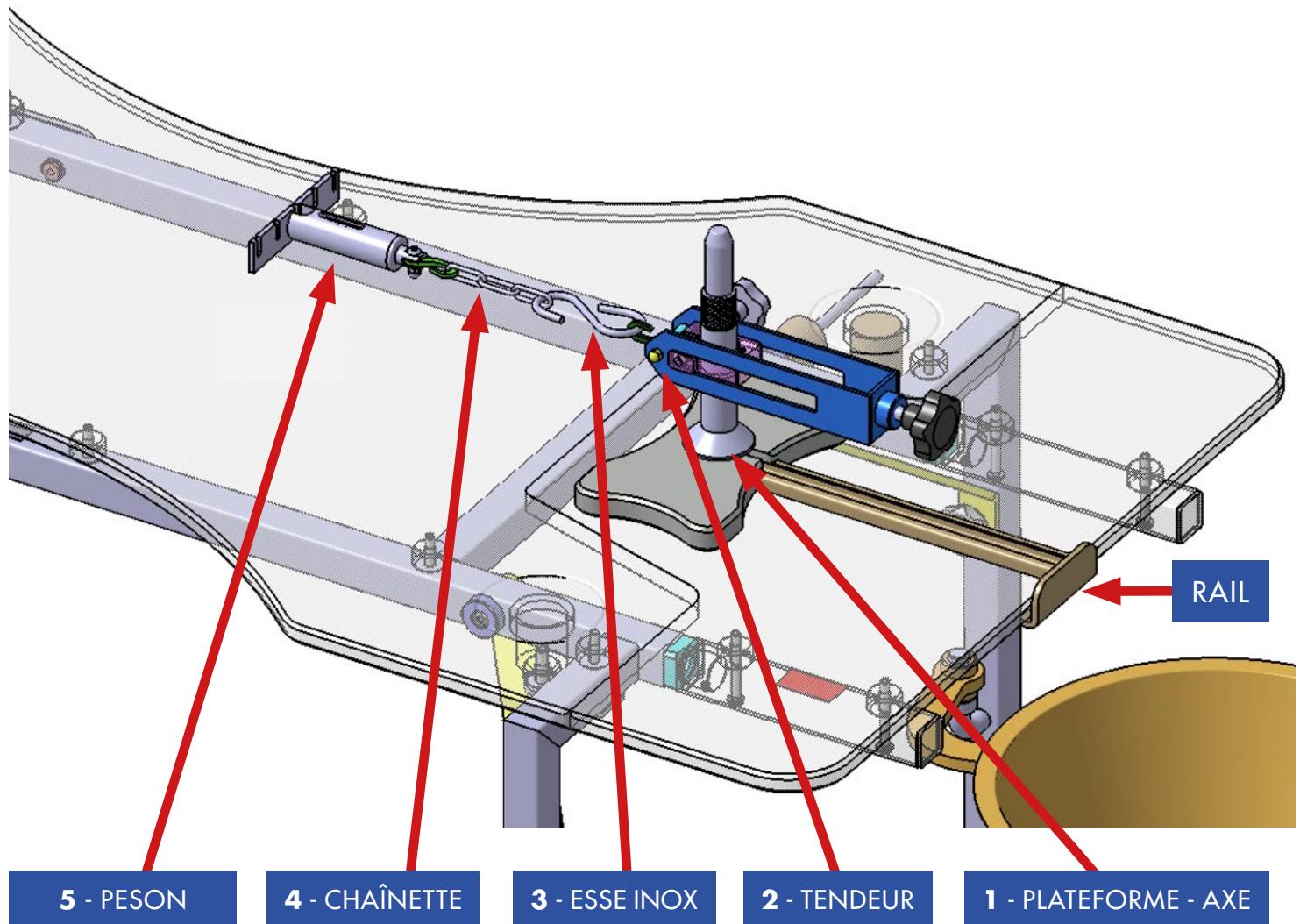
# Appendix 3 : Using horizontal traction
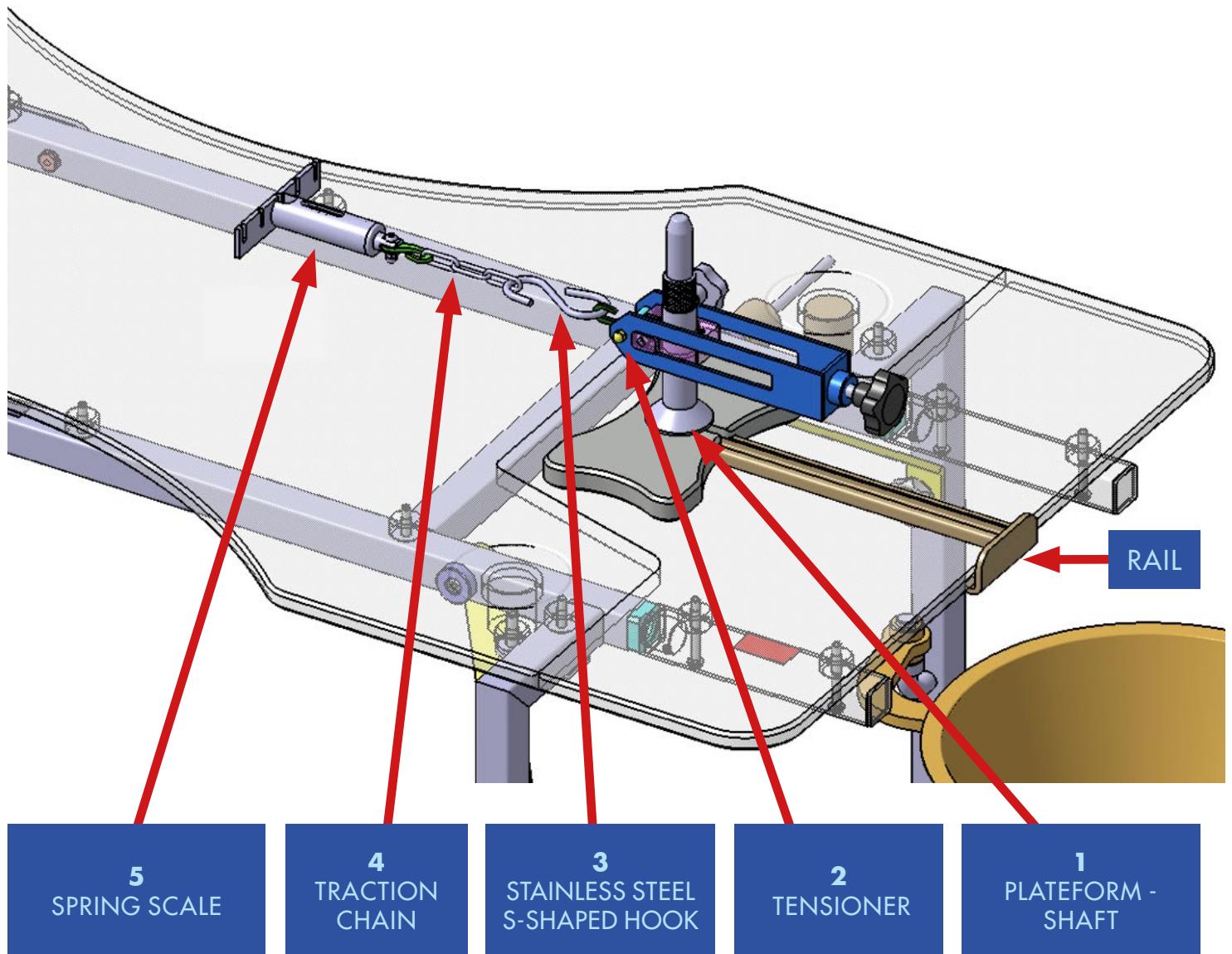
## A : SCHEMATIC WITH A HORIZONTAL VIEW


Horizontal traction assembly follows the identical order used for vertical installation:


- 1 - Rail-platform-shaft (2 model options)
- 2 - Tensioner
- 3 - Stainless steel S-shaped hook
- 4 - Traction chain (optional)
- 5 - Spring scale (2 model options)

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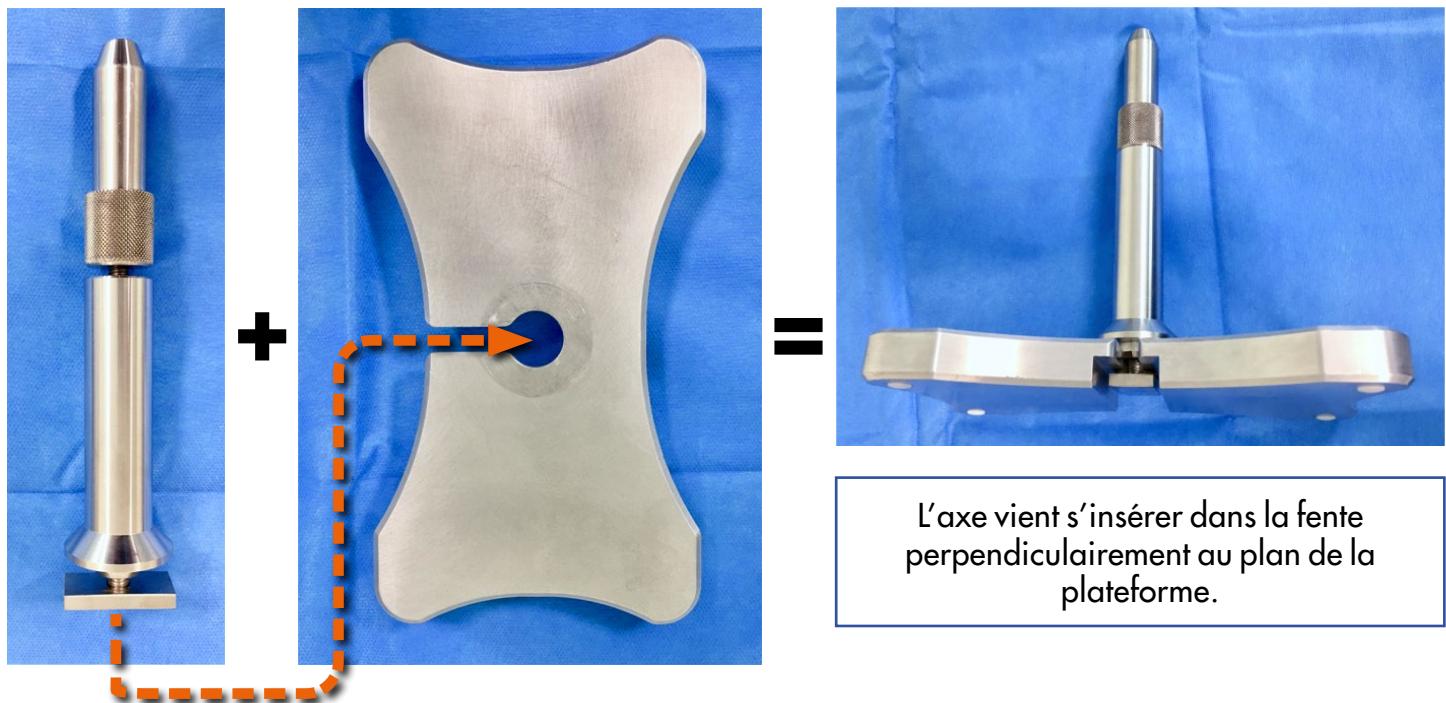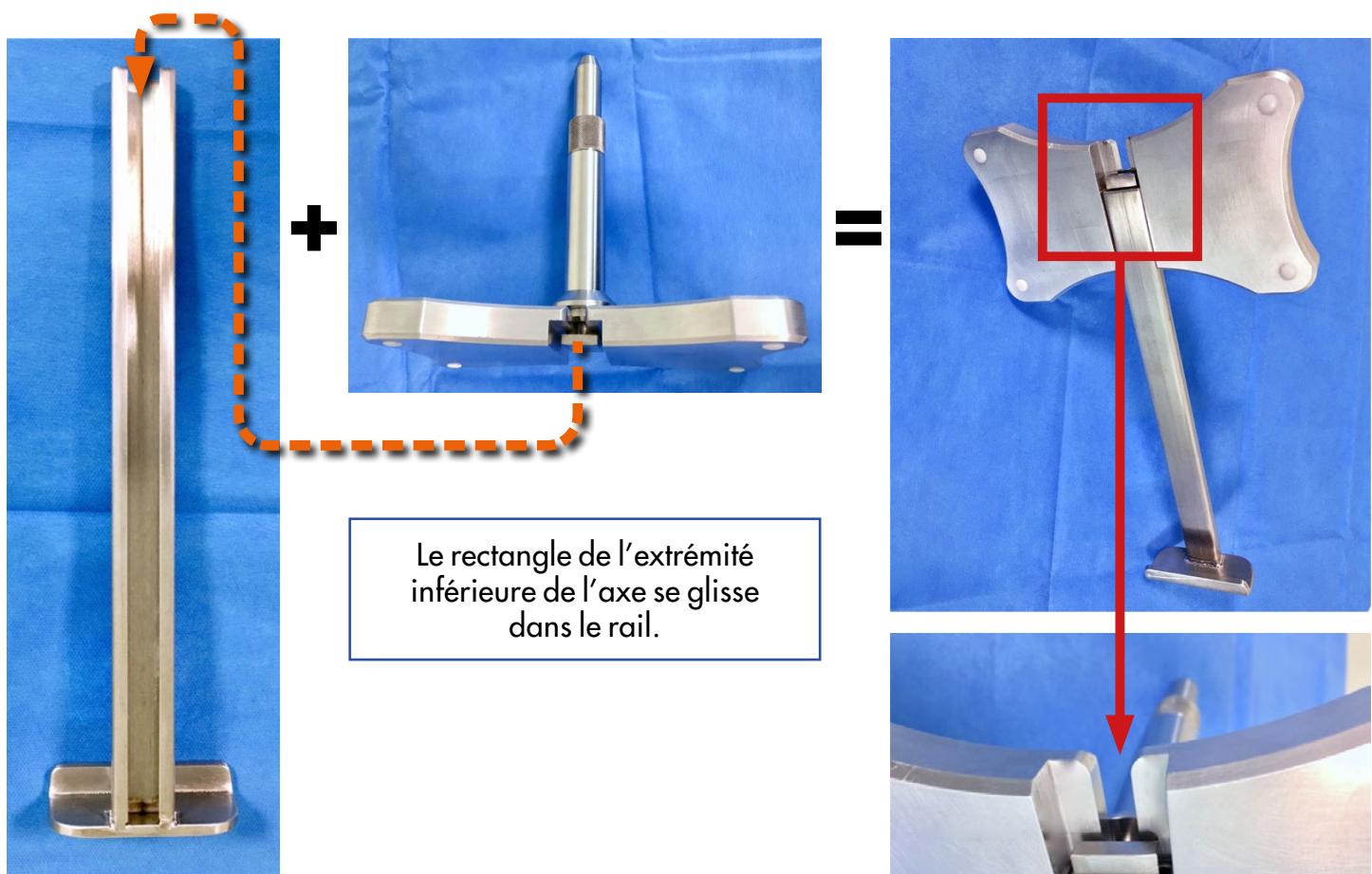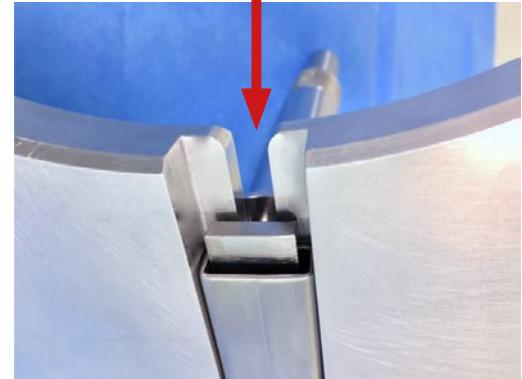
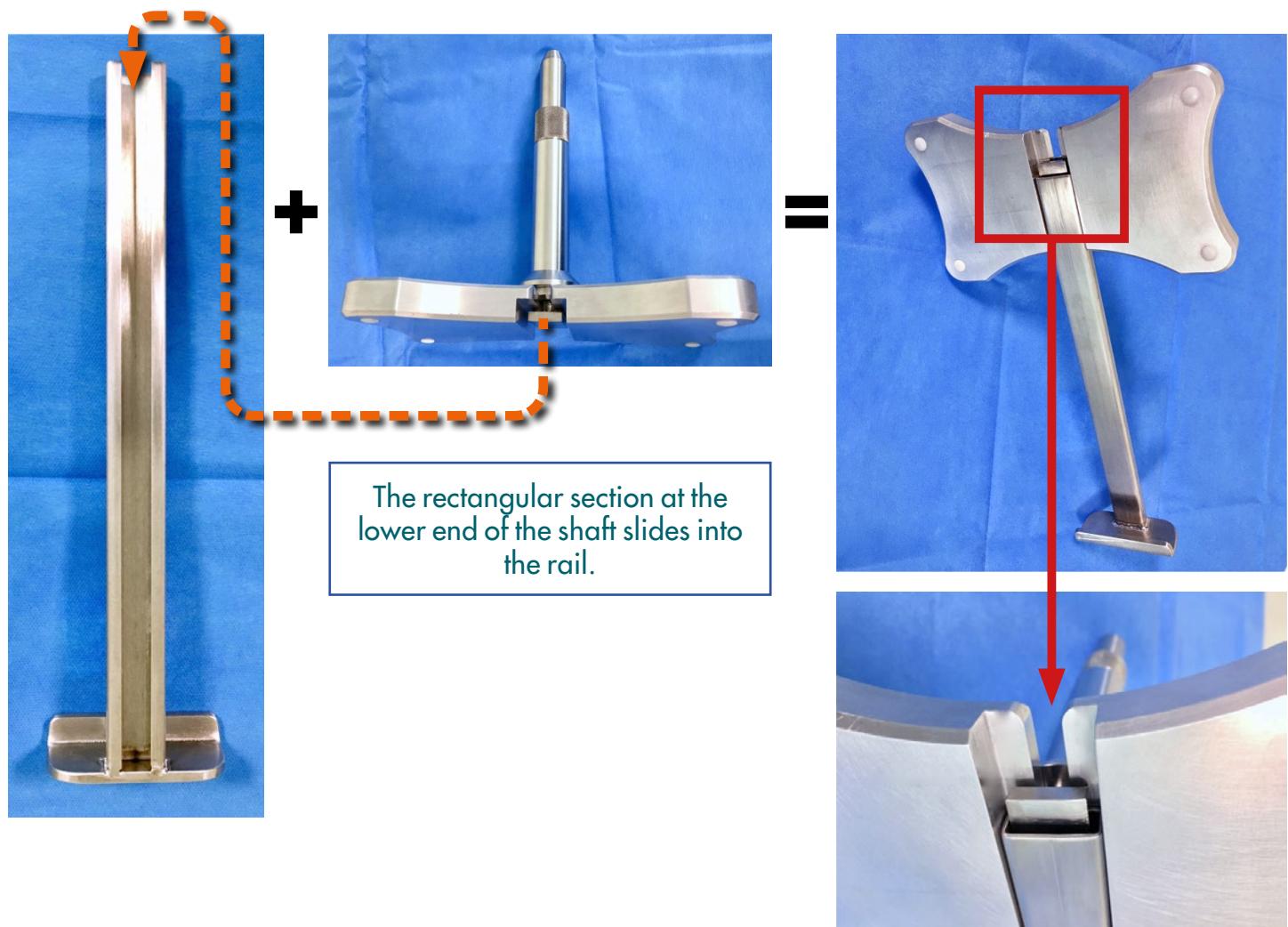
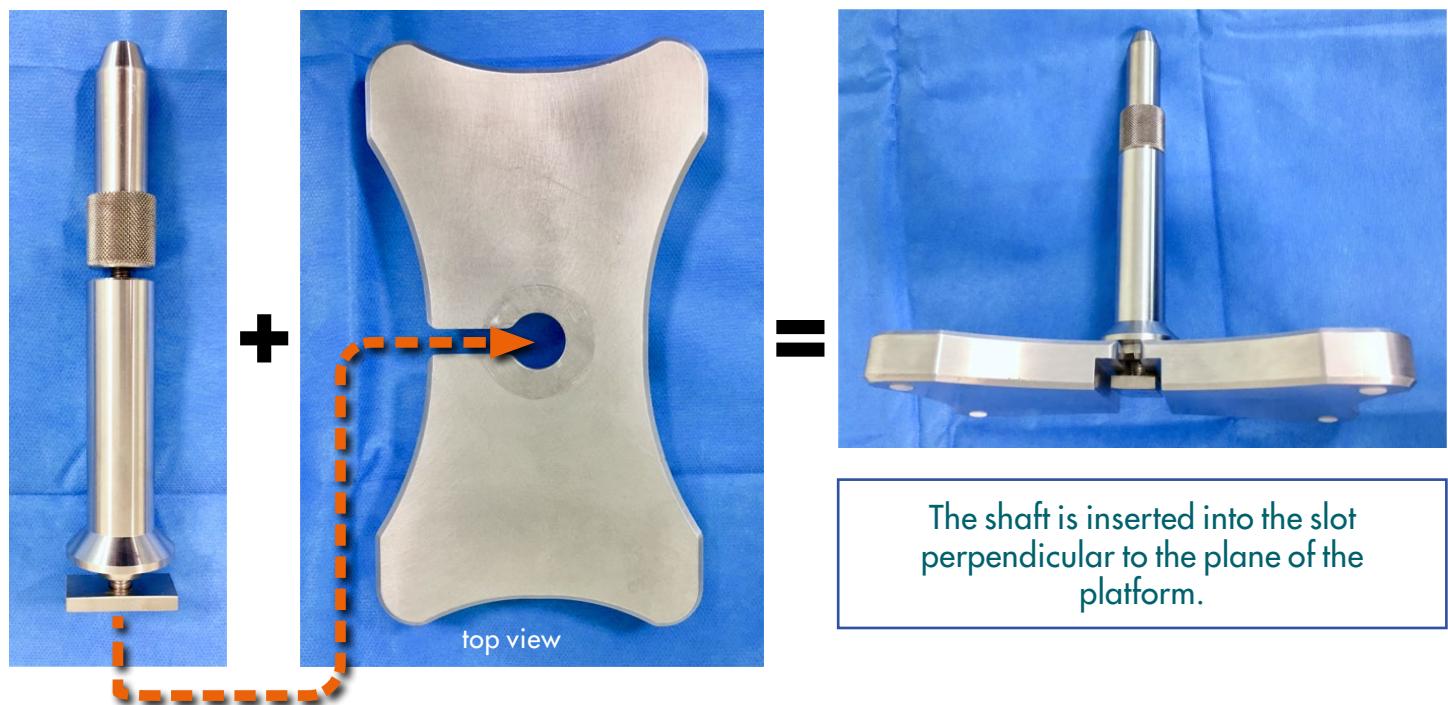
### MODEL 1: NON-STERILISABLE TRANSPARENT PLATFORM

A transparent, non-sterilisable platform designed for use in emergency department plaster rooms to carry out reductions followed by immobilisations under non-sterile conditions.

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### MODEL 2: STERILISABLE METAL PLATFORM

A sterilisable metal platform designed for use in operating theatres to carry out surgical procedures under sterile conditions involving traction of the upper limb (for example, a radial glenoid fracture). This unit may, of course, also be used in non-sterile settings.

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The shaft's outer diameter corresponds to the support rod's inner diameter, allowing one to fit into the other.

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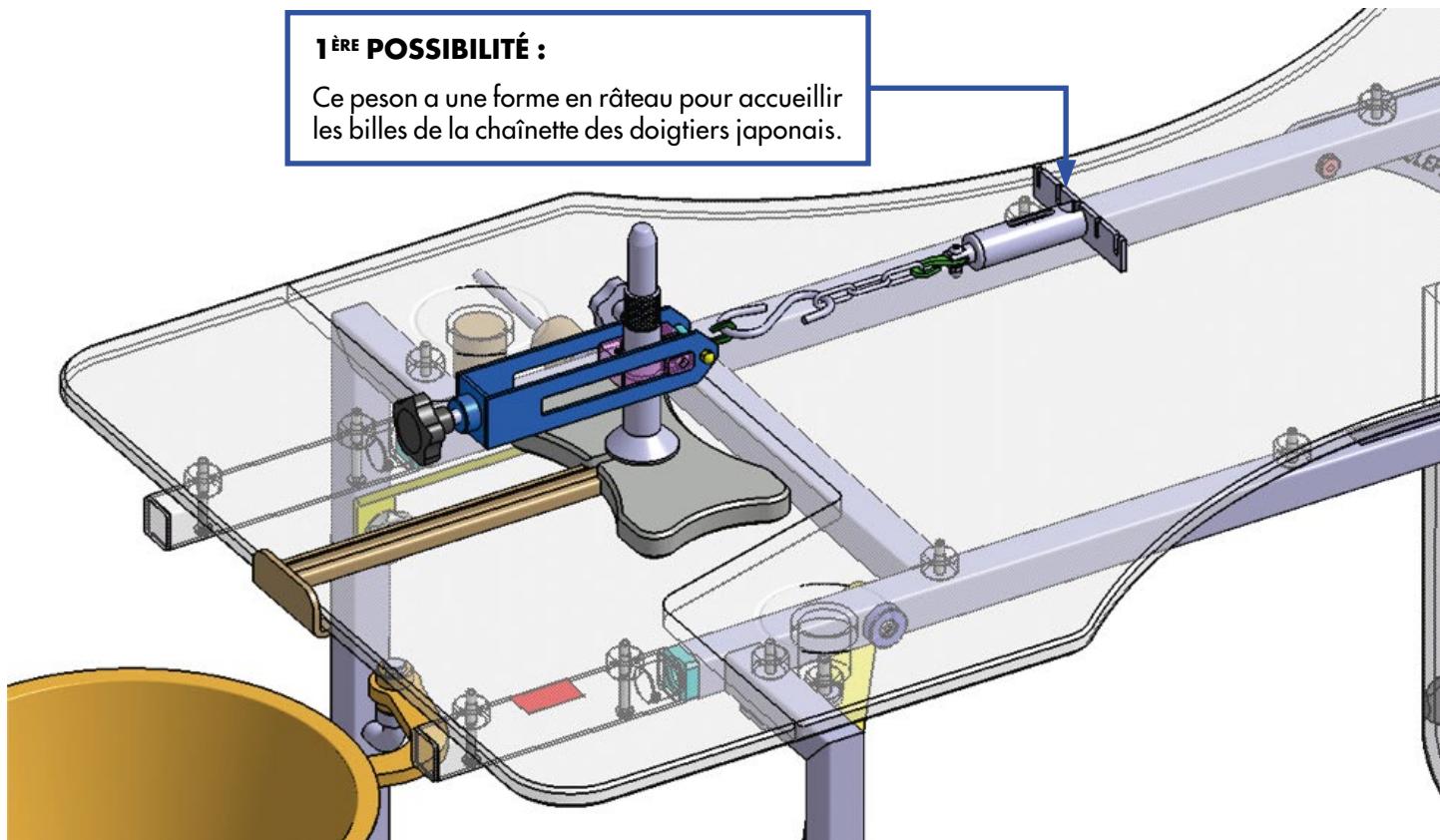
The knurled ring allows the assembly to be tightened and loosened.

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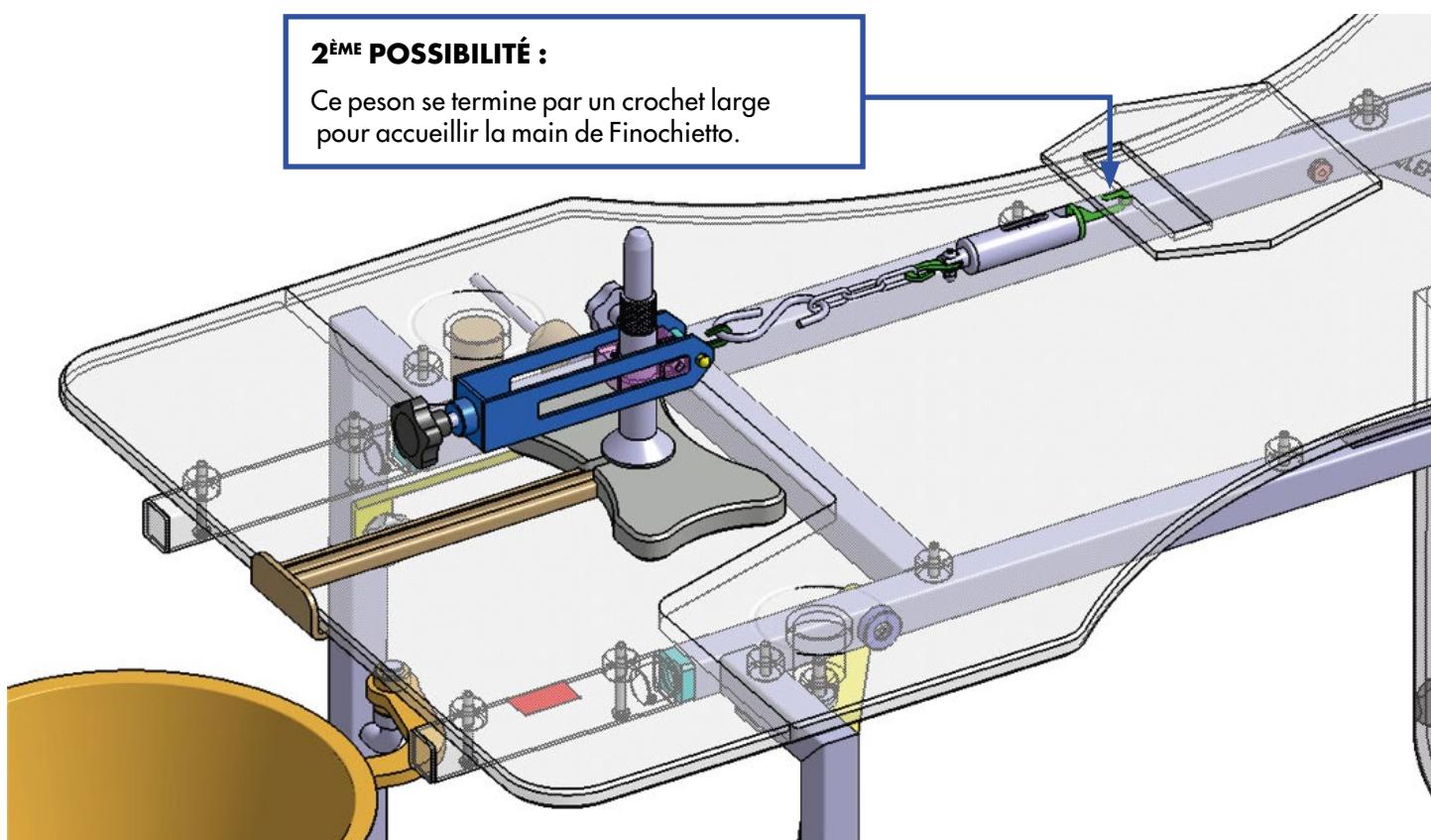
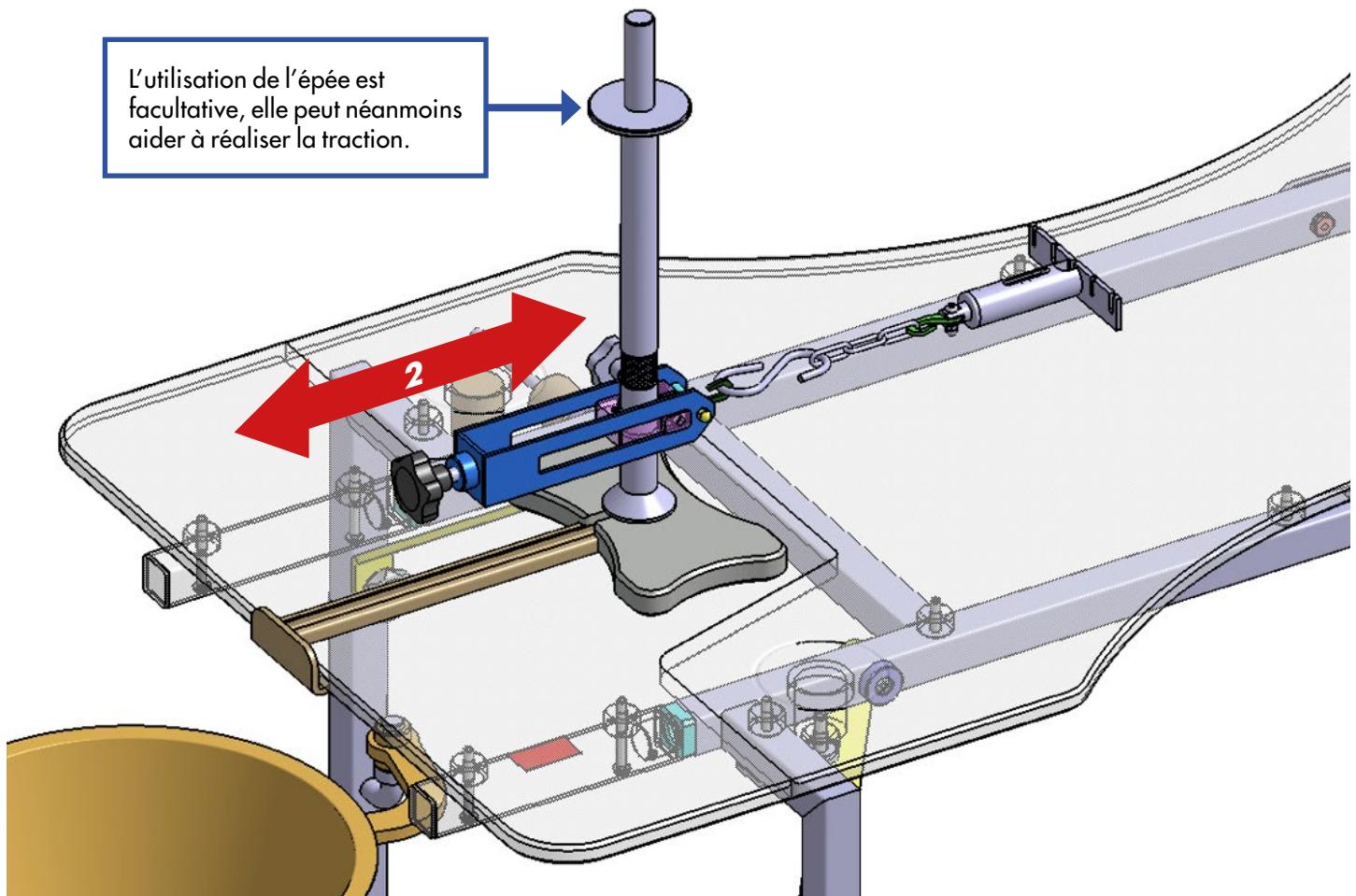
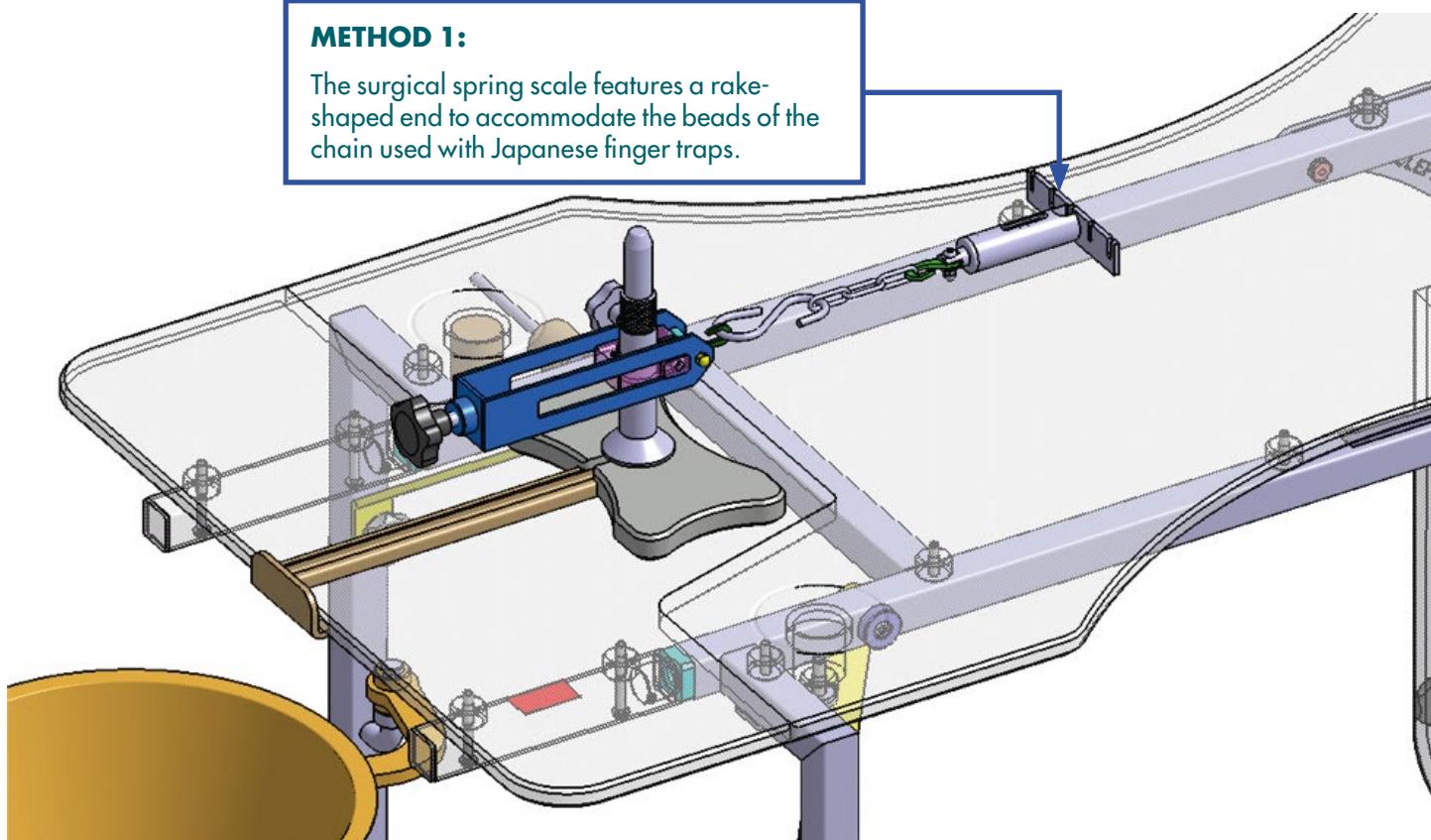
## Appendix 4 :

# Patient finger stabilisation: Japanese finger traps / Finochietto handle

The operator may choose between two patient finger fixation methods:


### METHOD 1:

The surgical spring scale features a rake-shaped end to accommodate the beads of the chain used with Japanese finger traps.


### METHOD 2:

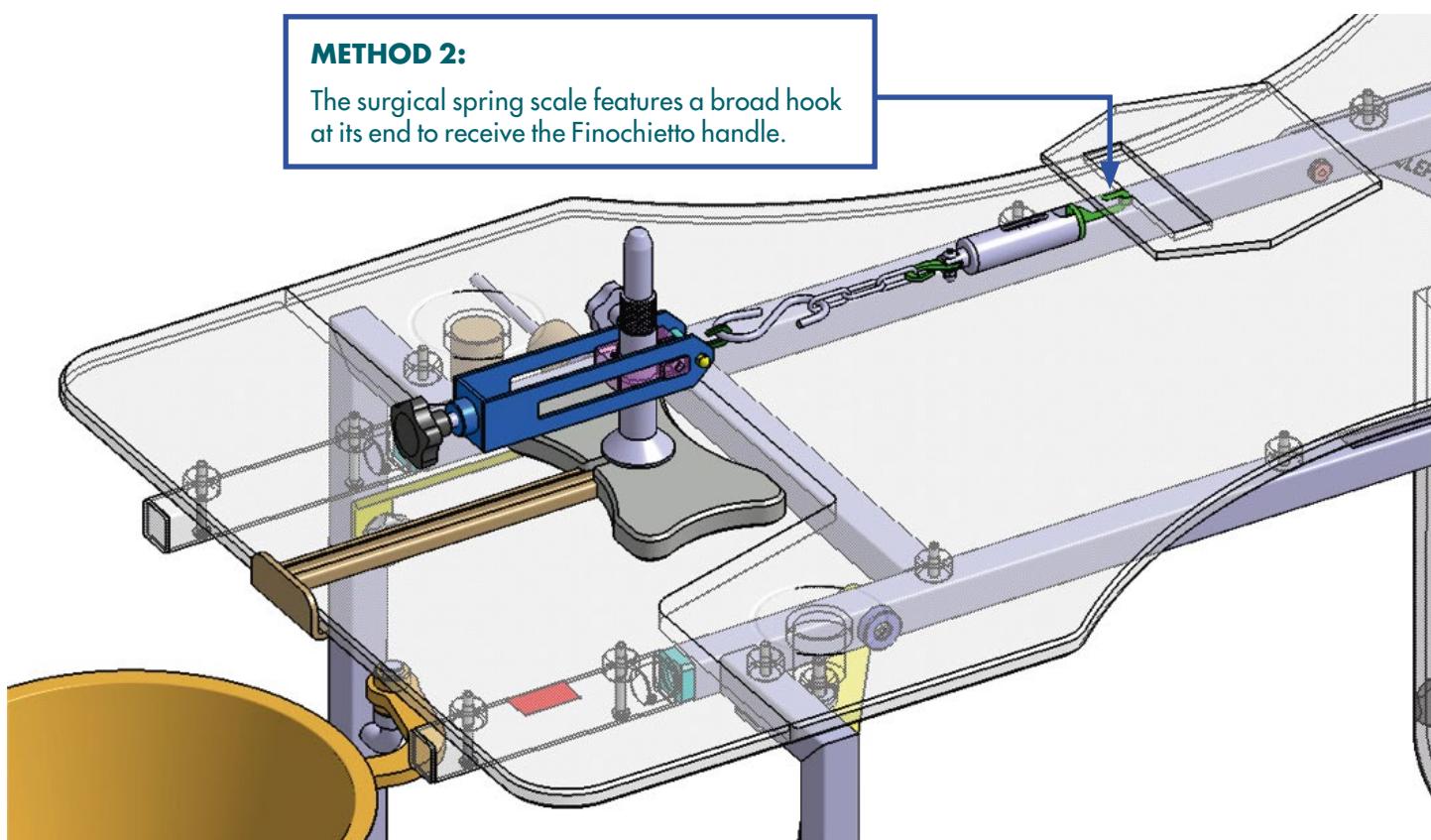The surgical spring scale features a broad hook at its end to receive the Finochietto handle.

# Installation – traction – locking:

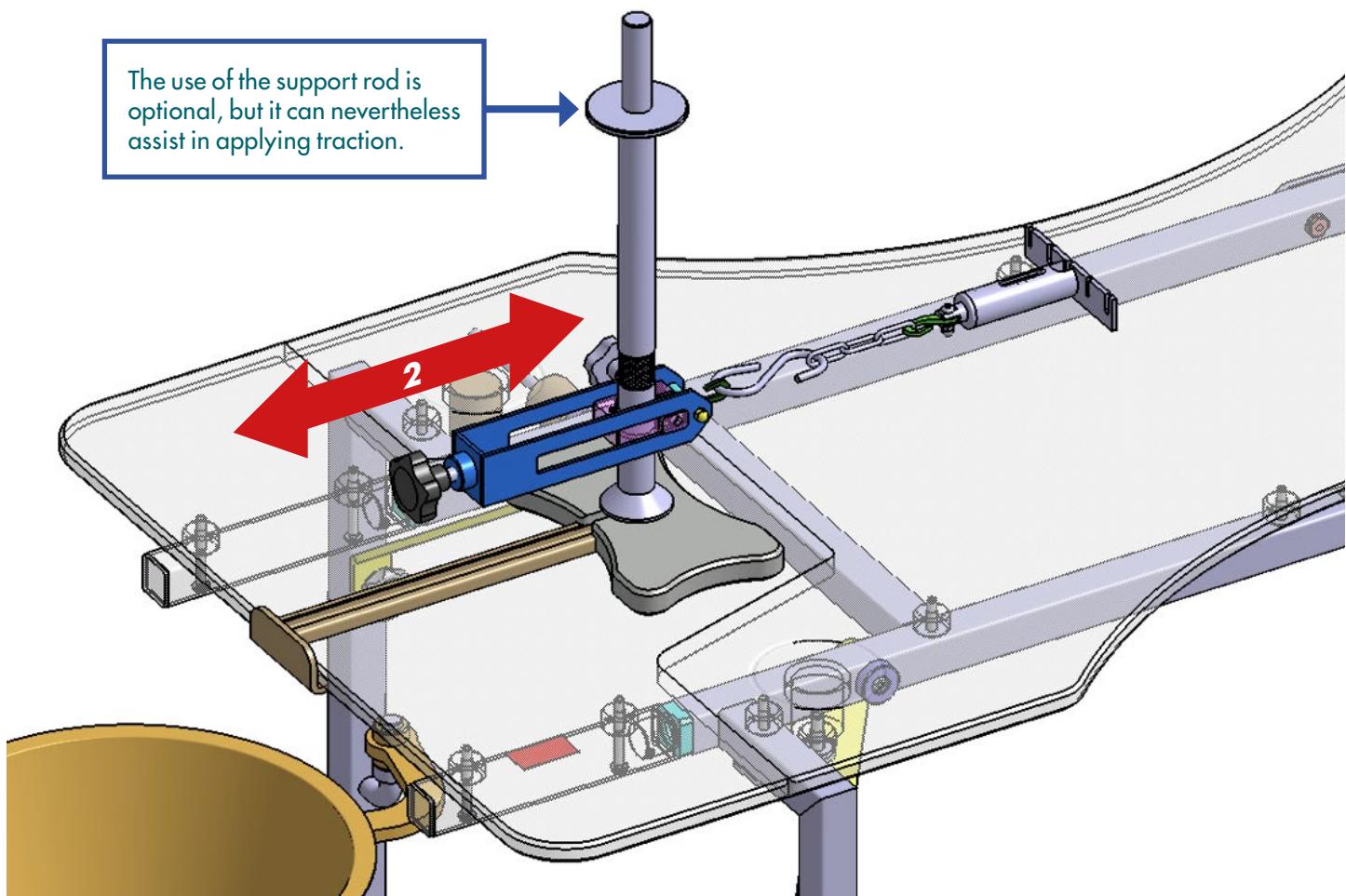The use of the support rod is optional, but it can nevertheless assist in applying traction.

**1**

## INSTALLATION

This is the first step: install everything without traction.

**2**

## TRACTION

Apply traction by loosening the knurled ring and drawing in line with the shaft's axis.

**3**

## LOCKING

Tighten the knurled ring to maintain traction.

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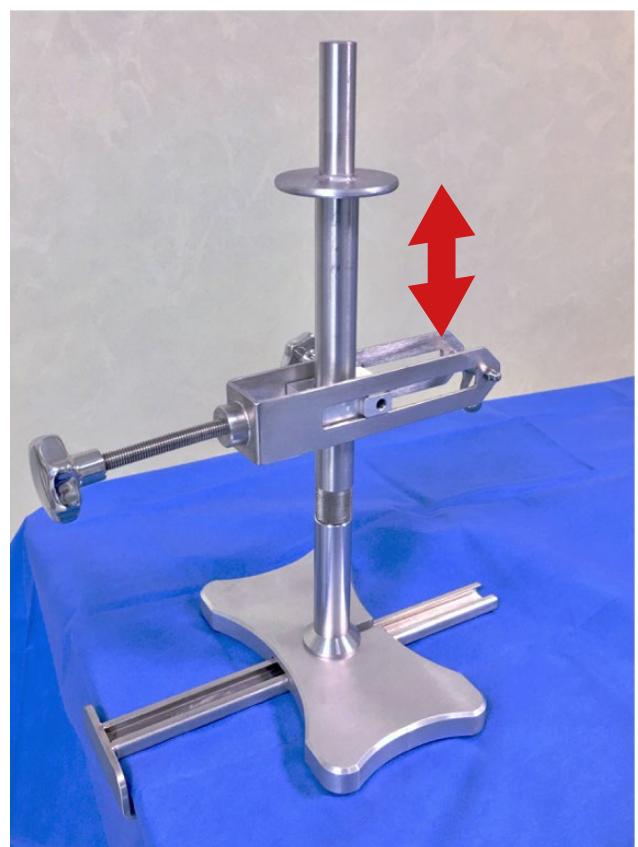
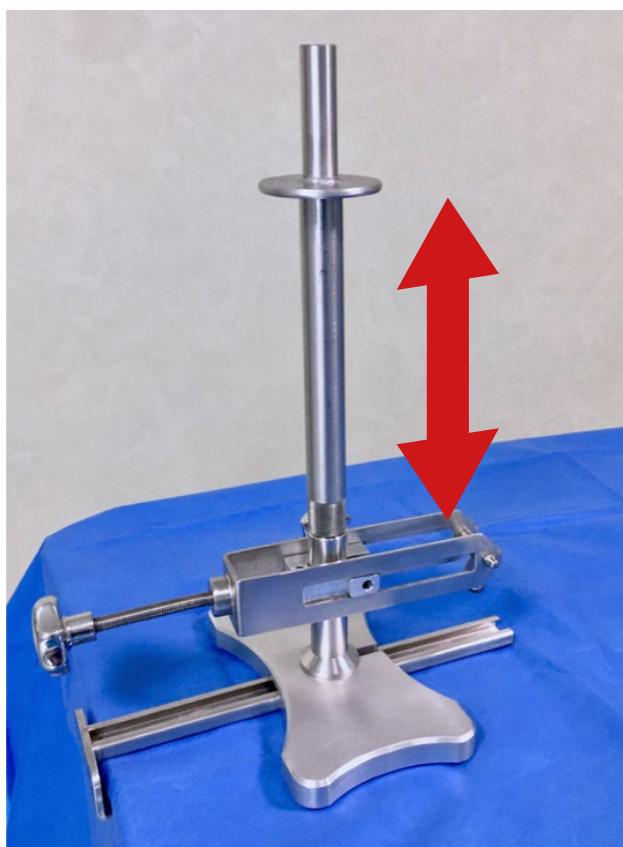
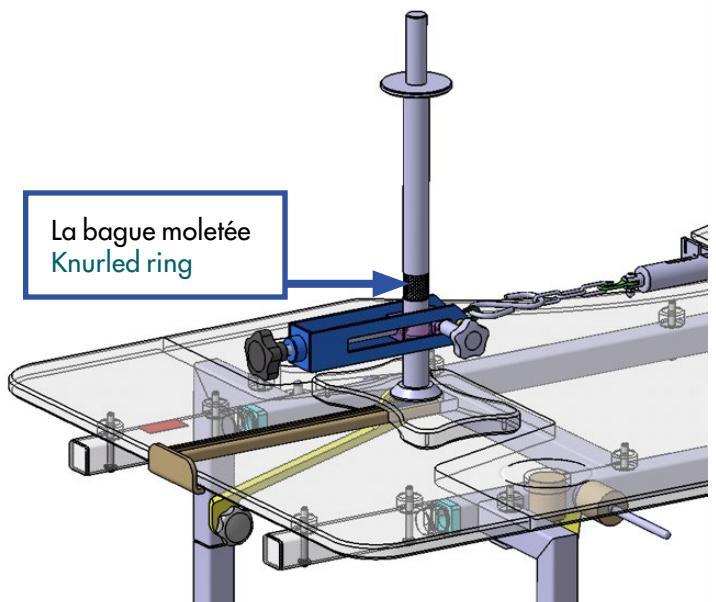
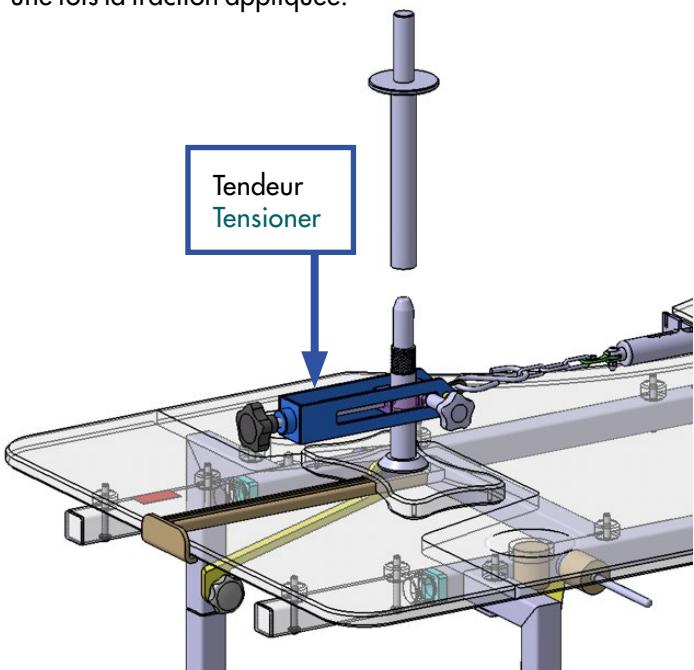
In order to clear the forearm and wrist circumference—for example to facilitate the moulding of a resin cast—the shaft can be extended by the arthroscopy support rod, as their diameters are compatible.

The tensioner can then slide up along the support rod's shaft to the guard.


Avoid positioning the tensioner at knurled ring level, since this must remain accessible to lock the system once traction is applied.

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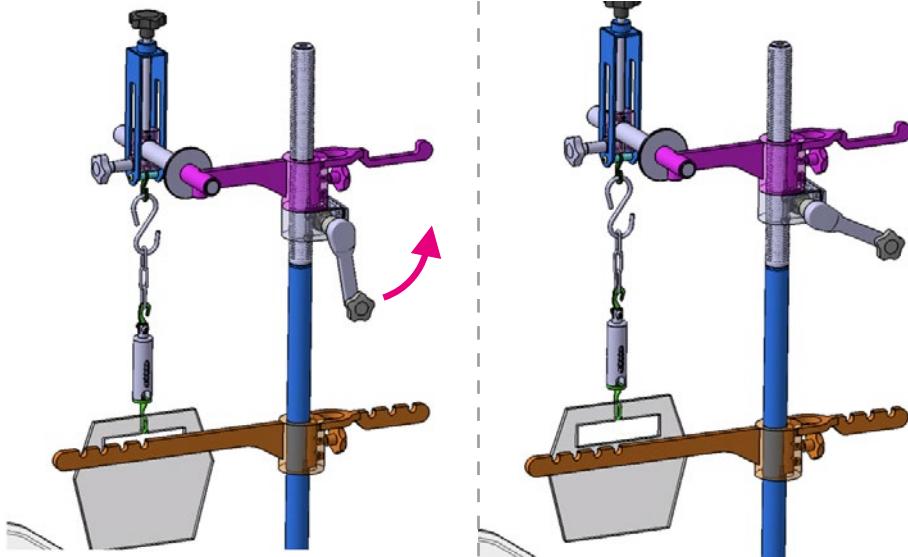The ratchet mechanism applies traction directly to the arthroscopy support.


The multiplied force lightens the effort for the circulating team.

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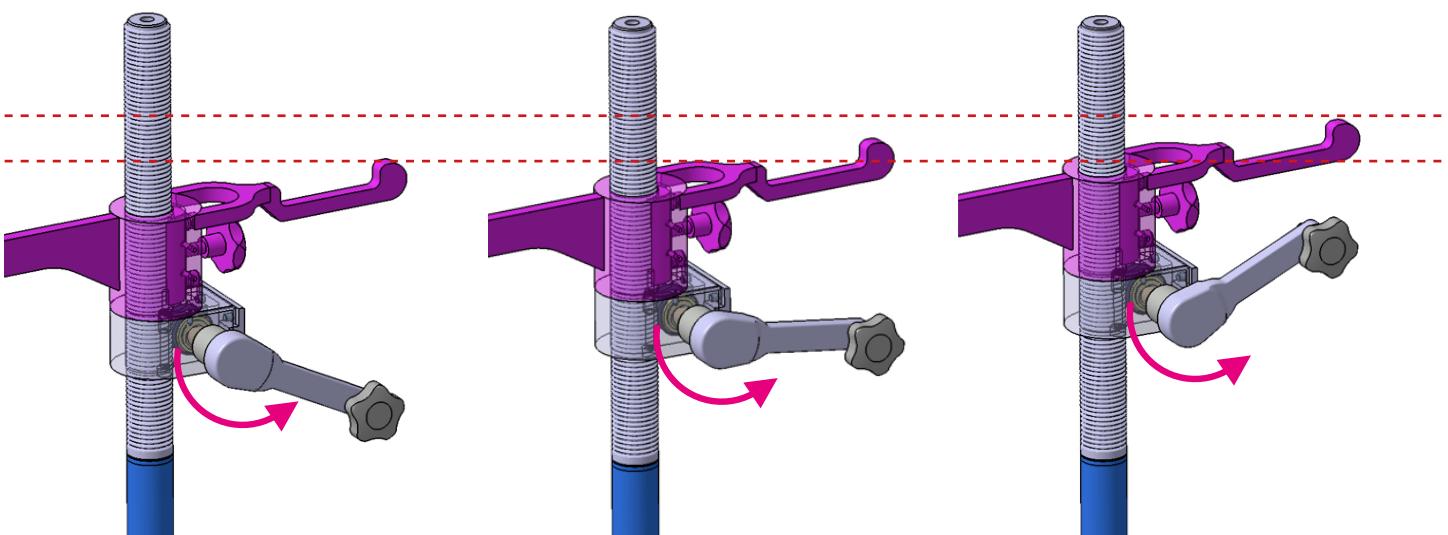Caution:  
**The traction generated is powerful!**

**Therefore, turn the ratchet mechanism sparingly and always monitor the traction force by checking the spring scale directly.**


## Annexe 7 / Appendix 7 :

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## Traction known as the “Alsatian model”

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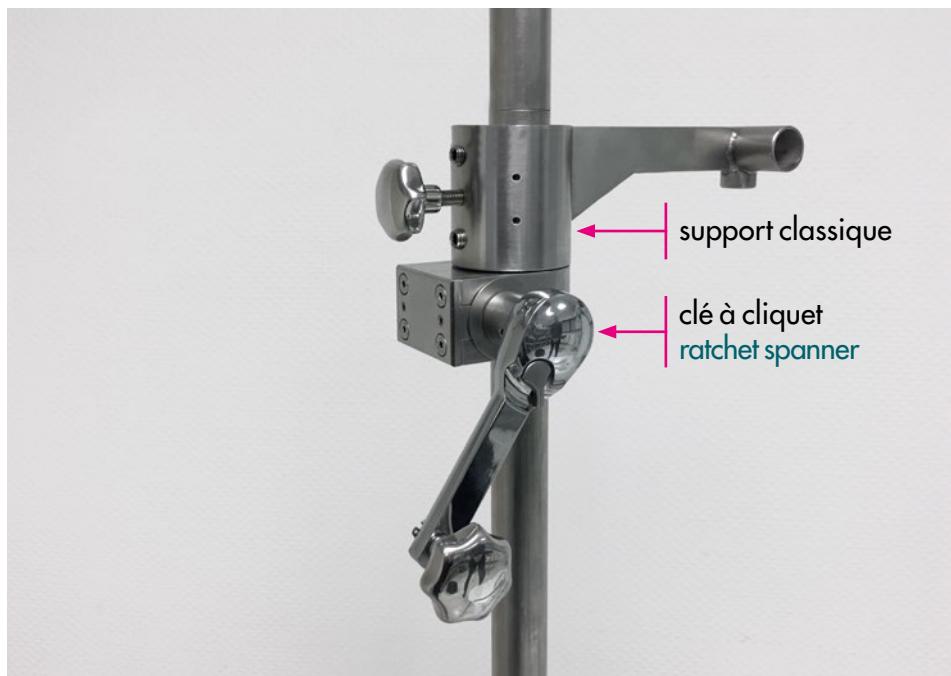Thanks to user feedback, there is now an assisted version of the support arm: a ratchet gear mounts beneath the arthroscopy support.

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Drawing on experience from the Strasbourg University Hospital teams, this system is driven by a ratchet gear operated by a ratchet spanner.

A ball detent secures the spanner in its square drive.

The traction block is symmetrical, with a square drive on each side, making the system fully reversible for right- or left-handed use.

To keep the ratchet spanner readily accessible, position it beside the patient.

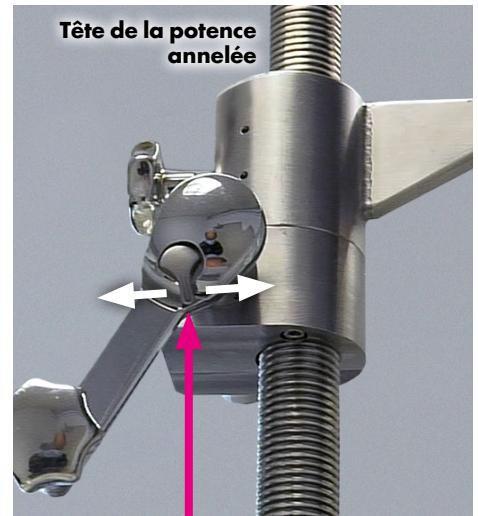
**PRECAUTION: Changing the side of the ratchet spanner must be done before patient setup.**

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To aid the circulating staff in applying this traction, the arthroscopy support can optionally be fitted with a force-multiplying pusher, or “winch.”

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**Caution: As with manual traction, the force applied must be strictly monitored on the spring scale's notches.**

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The ratchet's internal rotation direction can be reversed using a small switch.

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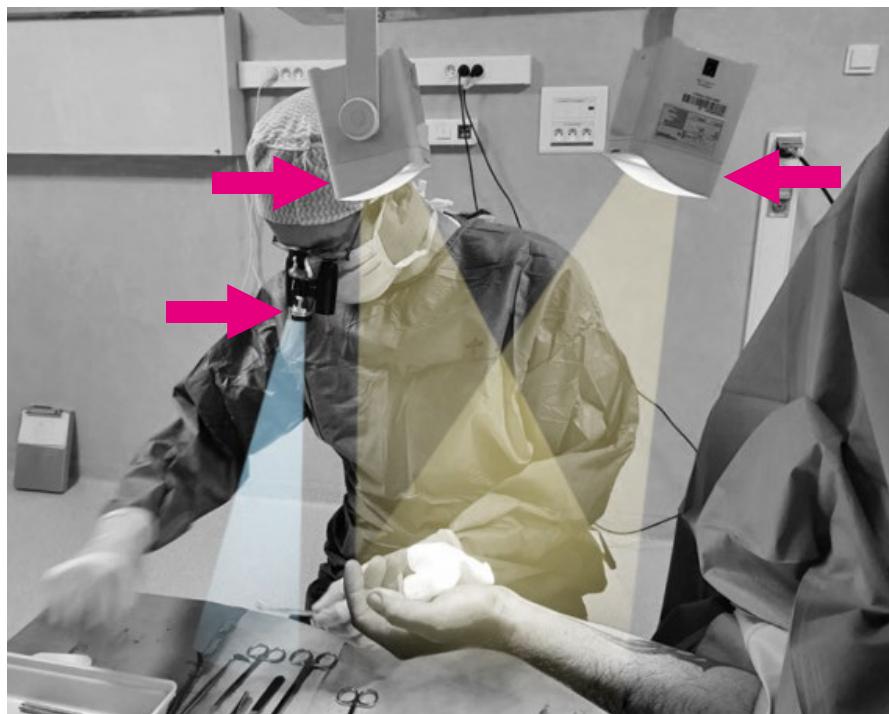
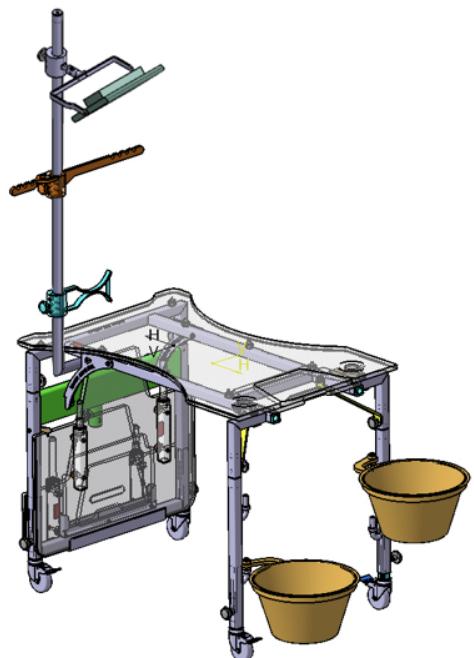
The recent adoption of the universal hand table in Office Surgery, and even as a consultation table, has prompted Asclepios to seek high-performance, mobile, autonomous (i.e. battery-powered) lighting solutions that attach to the mast's existing supports.

These zenithal operating-theatre lights are harmoniously complemented by an axial headlamp mounted on the operator's spectacles.

Several solutions are currently under evaluation at multiple centres; some examples are shown below.

This chapter is intended to evolve in line with feedback from practical experience.

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The orthotists at this centre chose to use a universal hand table in their dedicated room. Exceptionally ergonomic for orthosis fabrication, it here supports a simple articulated LED lamp with magnifier.

During routine consultations, the tabletop's cut-out shape helps the practitioner get closer to the patient's hand and elbow.

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Simplified universal hand table  
(for care and consultations)

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The consultation table was developed in response to a growing demand from surgeons and emergency physicians for a single piece of equipment in their consulting room that would **enable them both to conduct examinations and perform certain treatments.**

In other words, it serves as a consultation desk while also being usable, when required, for specific procedures, complete with a genuine work surface that meets the strictest hygiene standards.

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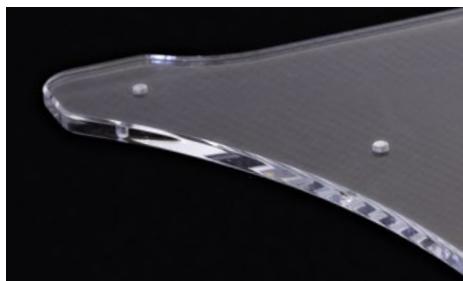
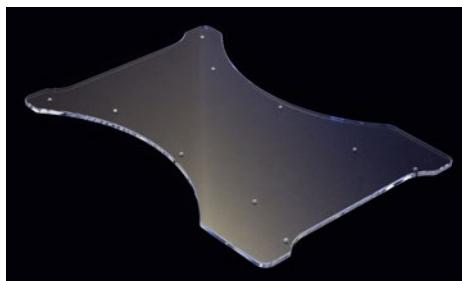
This new consultation hand table is built on the same solid and durable structure that underpinned the success of the universal hand table in the operating theatre.

Retaining the same design principles, it omits components exclusively needed for theatre use and instead incorporates features tailored to its new combined roles.

Modular by design, it benefits from the same spirit of universal compatibility, accommodating every existing and future option for the universal hand table.

This very mindset enables the apparatus to adapt as a practitioner's practice evolves throughout their career. For example, after several years of use as a consultation table, a clinician may choose to upgrade their table with additional options to begin performing certain procedures in their clinic.

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